Medela Breast Milk Storage Bottles, 2015

Medela Breast Milk Storage Bottles, 2015

Medela Breast Milk Storage Bottles, 2015

## Instructions for use Breast milk storage bottles

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ENGLISH
<div><div><span><span>⚠</span></span> Please read all instructions before using this product. Keep these instructions for future reference.</div></div>
<b>Intended use</b>
Medela breast milk storage bottles are intended to store, transport and warm human milk.
Important safety information
<div><div><span><span>⚠</span></span> <b>For your child's safety and health WARNING!</b></div></div> <ul style="list-style-type: none"><li>Keep components not in use out of the reach of children.</li> <li>Do not warm milk in a microwave oven as this may cause uneven heating and could scald the baby.</li> <li>Do not expose the product to direct sunlight or heat.</li> <li>Use only Medela original accessories to ensure the proper function.</li> <li>Use the product only for its intended use as described in these instructions for use.</li> <li>Other food heating, cleaning, storage and use methods have not been validated and should be avoided. It may damage the product and may present a health risk.</li></ul>
Cleaning and Disinfection

**Important**

- Only use drinking-quality water for cleaning and disinfecting.
- Before first use, clean and disinfect all parts (see chapter A & B).
- After each use, clean all parts (see chapter A).

- Once per day, after use, clean and disinfect all parts (see chapter A & B).

**Cleaning** – To clean the surfaces of the parts by physically removing milk residues and other possible contaminants.

**Disinfection** – To efficiently lower the number of germs that may be present on the surfaces of the washed parts.

**A. Cleaning**

- Disassemble the breast milk bottle into its individual parts.

- Rinse the disassembled parts with cold water (approx. 20 °C) immediately after use.
- Clean these parts with plenty of warm, soapy water (approx. 30 °C). Use a commercially available dish soap and clean soft brush.

- Thoroughly rinse all components again with cold water (approx. 20 °C).

- Dry the parts according to chapter C.

As an alternative to hand washing, in case of coarse dirt, rinse disassembled parts with plenty of cold water.

- At most once a day, place the disassembled parts on the top rack of a dishwasher.
- Use a commercially available dishwashing detergent.
- Dry the parts according to chapter C.

**B. Disinfection**

- Disassemble the breast milk bottle into its individual parts.

- Place the disassembled parts in a saucepan of sufficient capacity.

- Cover the disassembled, cleaned parts with plenty of water and boil for at least 5 minutes. (Ensure the parts are not in direct contact to the bottom of the saucepan.)
- Dry the parts according to chapter C.

As an alternative to boiling,

- Use Quick Clean™ Microwave bags in the microwave in accordance with the instructions on the bags.
- Dry the parts according to chapter C.

**C. Drying**

- Allow parts to air dry on a clean, unused dish towel or paper towel.
- Put the clean and dry parts in a clean and dust free storage bag or a clean environment. Do not store parts in an airtight container/bag if moist. It is important that all residual moisture dries.

Storing and thawing breast milk		
	<b>Storage guidelines for freshly expressed breast milk</b> (For healthy term babies)	
	<b>Room temperature</b> 16 to 25 °C	up to 4 hours is best
	<b>Refrigerator</b> at 4 °C or lower	up to 3 days is best
	<b>Freezer</b> -18 to -20 °C	up to 6 months is best
	<b>Breast milk thawed in the refrigerator</b>	At room temperature: up to 2 hours In refrigerator: up to 24 h max. Do not refreeze!

- Do not store breast milk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (this is at the back of the glass shelf above the vegetable compartment).
- You can freeze expressed breast milk in milk storage bottles or Breast Milk Storage Bags.
- Do not fill the bottles or bags more than 3/4 full to allow space for possible expansion.
- Label the storage bottles or Breast Milk Storage Bags with the date of expression.
- For more information on storing your breast milk, visit https://www.medela.com/en/breastfeeding-pumping/articles/pumping-tips/how-to-store-freeze-and-thaw-breast-milk

⚠ **Caution**

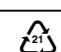
Do not thaw frozen breast milk or do not warm breast milk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

- In order to preserve the components of the breast milk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the storage bottle or Breast Milk Storage Bags under warm water (max. 37 °C).
- Gently swirl the storage bottle or Breast Milk Storage Bags to mix any fat that has separated. Avoid shaking or stirring the milk.
- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.

**Material:** Bottle, Lid: Polypropylene

Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.

**Disposal:** According to local regulations.

	Keep away from sunlight.		The packaging contains products intended to come in to contact with food according regulation 1935/2004.
	Keep dry.		Fragile, handle with care.
	Read and follow the instructions for use.		Indicates that an item can technically be recycled.
	Indicates the material from which an item is made.		

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