DE WELLNOISE Vorteile

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Schalldämmung SNR: 23 dB

GB WELLNOISE advantages

- malleable to suit the anatomy of the ear
- gentle on the skin and non-irritating
- high degree of noise reduction
- stays securely in place for the whole night and during sports activities
- protects against water splashes
- no uncomfortable pressure in the ear
- allergy tested
- recommended by ENT doctors

Non-compliance of these instructions at one's own risk. Non-compliance may severely diminish the effectiveness of the product.

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GB Reusable silicone ear plugs, tested to standard EU 2016/425

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GB Directions for use

3 pairs of earplugs made of silicone

1. Knead the WELLNOISE ear plug into a ball. Avoid oblong shapes (see picture).
2. Gently press dry and clean ear plug into the ear in front of the auditory canal.
3. To remove the WELLNOISE ear plug, gently pull on the auricle or earlobe. Remove only in one piece and do not use any other instruments.

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Important recommendations

This product is not made for permanent use. It is possible to clean the ear plugs with mild soap and warm water. Keep WELLNOISE ear plugs clean and dry in their box. Replace if the ear plugs are soiled and non-malleable due to use. Do not reduce. Also available: WELLNOISE ear plugs for children. Keep away from little children (risk of suffocation). In case of difficulties please contact an ENT doctor.

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WELLNOISE ear plugs protect your ears from

- noise: snoring, loud music, machine noise, street noise, aircraft noise ...
- water: swimming, sailing and similar water sports, showers and after operations on the ear ...
- cold: skiing, snowboarding, other winter sports ...
- wind: hiking, mountaineering, open-air sports activities ...

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