ManuBasic® Sport Handgelenkbandage

Wrist support | Bandage de poignet | Muñequera | Polsiera

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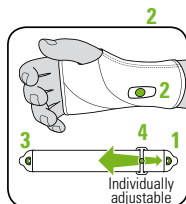
The **wrist joint support ManuBasic® Sport from BORT** supports damaged tissue structures (joints, ligaments, capsules and cartilage). Here, the compression strengthens the retaining function of the ligaments, thus relieving the joint. It can be individually adapted with the elastic **Stabilo®** strap with the easyClick system and the thumb hollow hinders the support from slipping. Thanks to the highly elastic **SOFTFlex 3D** knitted fabric, the support is easy to put on and nonetheless achieves a good supporting effect.

COOLMAX® fibre wicks moisture away from the body, ensuring dry and pleasant wearing comfort.

Instructions for use:

Adaptation of the anatomically pre-shaped plastic splint: If required, the material of the plastic splint can be individually adapted by the orthopaedic technician at a temperature of about 180 °C. Allow the reshaped splint to cool and then re-insert into the splint pocket of the support.

Place the support on the metacarpophalangeal joint affected and guide the thumb through the intended thumb opening at the same time.



Stabilo® strap with the easyClick system for additional stabilisation of the wrist:

1 and **3** attachment module

2 supporting module

4 adjustment module

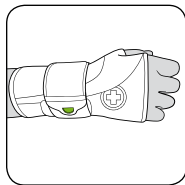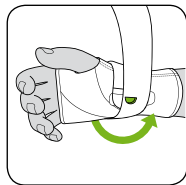
Adapt the stabilisation degree of the Stabilo® strap by moving the adjustment module (**4**) individually.

When you receive your support, the attachment Stabilo® strap module (**1**) is attached to the support in the supporting module (**2**).

Guide the Stabilo® strap circularly around the wrist under slight tension.

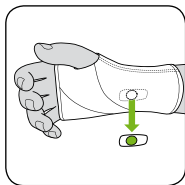Close the strap by attaching the free attachment module (**3**) to the adjustment module (**4**). Pay attention to comfortable fitting of the Stabilo® strap, neither too loose nor too tight.

If the Stabilo® strap with the easyClick system is not needed, the supporting module (2) can be taken from the velour bag. For this purpose, slide the module out of the opening in the velour.

Indications:

Chronic, post-traumatic or post-operative irritations in the wrist area, e.g. distortion, carpal arthrosis, tenosynovitis

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

40 % polyamide (COOLMAX®), 40 % polyamide, 19 % elastane (LYCRA®), 1 % silicone

Important Note:

No re-use – this medical aid is intended for treating one patient.

If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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Removing the plastic splint, wash by hand in hand warm water with light-duty detergent. Fix the Velcro closure to avoid damage to other laundry. Pull into shape and air-dry. Avoid heat.

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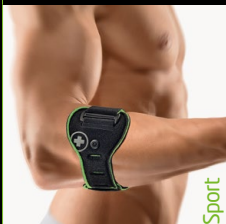
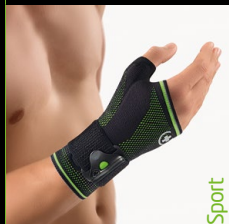
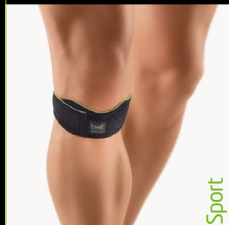
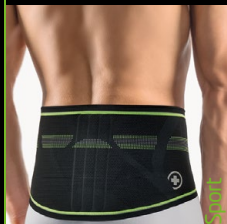
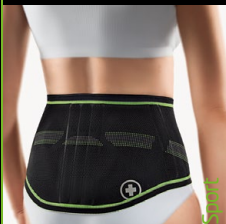
EN The **BORT ManuBasic® Sport** stabilises and supports the wrist whilst not impeding everyday life or sports. The integrated viscoelastic pad reduces pressure on the nerves and vessels. It combines the wearing comfort of a traditional hand joint support with the moisture regulating properties of the high-tech **COOLMAX®** fibre. The **SOFTflex 3D** knitted fabric ensures elasticity, stability and non-slip fit with every body movement. The **Stabilo®** strap with the innovative **easyClick** system provides additional stabilisation.

BORT Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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