Instructions for Use

Cutimed® Sorbion® Sana (EN)

Product description
Atraumatisk, superabsorbent dressing Cutimed® Sorbion® Sana er en steril, hydroactit, gel-forming wound dressing with strong exudate absorbing ability. The atraumatic dressing consists of a water-retaining superabsorbent core, an in-between layer of polyhydrogel polypropylene and an outer wound contact layer made of polyethylene film with three-dimensional perforations. Both sides of the dressing can be applied to the wound. The product does not contain any pharmacologically active substances and is free of binders or adhesives.

Characteristics
Cutimed® Sorbion® Sana absorbs and reliably retains wound fluids within the superabsorbent core even under compression. This leads to a removal of wound healing inhibiting factors, including bacteria, proteases and inflammatory mediators and minimizes the risk of leakage and maceration. At the same time, the dressing achieves a moist wound environment, which supports the body's own wound healing.

The high absorption capacity of Cutimed® Sorbion® Sana enables prolonged wound rest compared to conventional wound dressings. Its enhanced fluid handling capacity reduces unpleasant odors. Non-viable tissue adheres to the dressing and is gently removed at dressing change, supporting soft debridement and reducing the risk of infection. Due to its 3-D-structure and smooth surface the outer wound contact layer facilitates the fluid flow in the direction of the dressing, while offering few points of access for ingrowing tissue, consequently, providing a protective layer against adherence to the wound bed. Cutimed® Sorbion® Sana is particularly suitable for wounds with a heterogeneous healing process by ensuring atraumatic dressing change.

Intended Purpose

Cutimed® Sorbion® Sana is intended for the treatment of wounds with low to excessive exudate levels, such as decubitus ulcers, ulcus cruris, diabetic foot ulcers and other wounds healing through secondary intention, dehisced wounds, as well as after burns, on donor sites and fragile skin, epidermolysis bullosa (EB) and hidradenitis suppurativa (HS).

Please contact your healthcare professional if you are unsure whether the product is appropriate for you.

Contraindications

There are no known contraindications.

Do not use if the pouch is already open or damaged as the sterility of the dressing is guaranteed only when the pouch is unopened and undamaged prior to use. Cutimed® Sorbion® Sana is for single use only. Re-use may lead to an increased risk of infection or cross contamination. Do not re-sterilize. Cutimed® Sorbion® Sana should be used under medical supervision. Please always follow the instructions of any product that is used in combination. Do not cut the dressing. It must remain intact and uncut. Avoid contact with the following areas: mucous membranes (risk of drying and adhesion), eye area.

Cutimed® Sorbion® Sana should not be used in combination with therapies involving increased heat development or in which high-energy radiation (UV, V- or x-ray) is used, since this can compromise the mechanical integrity of the product.

Only apply to tunnel building pockets such as decubitus ulcers under special supervision. The product expands as it absorbs exudate and may adversely affect the dressing under certain circumstances. Cutimed® Sorbion® Sana can cause products display high affinity to fluids. This may result, if this product is not used as intended (e.g. prolonged use), in adherence to the wound or fusion with the surrounding tissue, despite its atraumatic properties.

If the treated condition deteriorates, fails to improve or if a side effect is observed, consult a physician or an appropriate health care professional. A temporary increase in wound size may occur at the start of a normal healing process; consult a medical professional to determine the cause of any significant change in wound size, color, or odor.

Combination with other wound care products

Cutimed® Sorbion® Sana should not be used in conjunction with petroleum or grease based products, ointments or creams, as this may obstruct the pores of the dressing and reduce its efficacy.

Directions for use

Application and removal of the dressing should be in accordance with good clinical practice.

- Preparation:**
- Before applying the dressing disinfect hands and use gloves.
 - If necessary, irrigate the wound and remove leftover cleansing solution afterwards.
 - Select a suitable dressing size, so that the absorbent inner core overlaps the wound margins by at least 1 cm/0.4-inch.
 - When taking the product out of the pouch, do not touch the wound pad.
- Application:**
- Apply the dressing towards the wound. Both sides of the dressing can be applied to the wound.
 - Secure the dressing after selecting an appropriate fixation for the condition of the wound surroundings and the skin. Films, medical tapes, bandages or any other suitable fixation dressing can be used. While fixing the dressing, take expansion during absorption into consideration.

Combination with other products:
Cutimed® Sorbion® Sana may be used on infected wounds in addition to an antimicrobial therapy. If necessary, Cutimed® Sorbion® Sana can be used as secondary dressing in combination with other dressings, such as:

- a. with Cutimed® Sorbact®/Sorbact® in case of infected wounds,
 - a. with a gel-based product on necrotic wounds (e.g. Cutimed® Gel).
- Dressing change:**
Cutimed® Sorbion® Sana can remain in place for up to 7 days, depending on wound conditions and saturation of the wound dressing. It is recommended to change the dressing every 24 hours initially, moving to less frequent changes as necessary. A completely soiled dressing must always be avoided.

Removal:
To remove Cutimed® Sorbion® Sana remove fixation tapes, lift one corner of Cutimed® Sorbion® Sana, gently remove the dressing and discard. Adherence of wound deposits to Cutimed® Sorbion® Sana is a desirable effect, contributing to wound cleansing (soft debridement).

Storage and Disposal

Store dressing away from direct sunlight at ambient temperature and humidity. Disposal should be made according to local regulations, e.g. via regular household or clinical waste.

Notice

Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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