Instructions for Use

**Cutimed® Alginate**

EN

**Ingredient**

Mainly calcium alginate fiber

**Product description**

Cutimed® Alginate is a sterile, non-medicated alginate dressing, consisting of mainly calcium alginate fiber. Cutimed® Alginate is made of natural fibers derived from seaweed. A hidden integrity layer in the dressing gives it high wet stability. It is designed to be a highly, fast absorbent and soft, sterile non-woven dressing. Cutimed® Alginate absorbs wound exudate and forms a gel-like covering over the wound, maintaining a moist wound healing environment which accelerates the healing process and helps to stop bleeding. The gel-like surface also keeps the dressing from adhering to the wound and enables a painless removal. The high wet strength of Cutimed® Alginate ensures the dressing remains integral while removing.

**Intended purpose**

Cutimed® Alginate is indicated for the management of moderate to highly exuding partial to full thickness wounds, including:

- Surface trauma
- Cavity wounds
- Pressure ulcers
- Diabetic foot ulcers
- Venous/arterial leg ulcers
- Donor sites
- Post-operative wounds

Cutimed® Alginate dressings are suitable for use under compression bandages.

**Contraindications**

Cutimed® Alginate dressing should not be used on individuals who are sensitive to or who have had an allergic reaction to alginate or polyester.

**Precautions and Observations**

- This product is for single use.
- Do not use if package is damaged.
- The dressing is used on infected wounds only under the care of a healthcare professional.
- The dressing is not indicated for surgical implantation and dry wounds.
- The dressing is not recommended to control heavy bleeding.
- If redness or allergic symptoms appear, please stop using this product.
- Single product cannot stay on wound for more than 7 days.
- The dressing is not recommended for infants less than 12 months of age or pregnant woman.

**Directions for use**

Application and removal of the dressing should be in accordance with good clinical practice.

- Clean the wound with sterile saline water and dry the surrounding skin gently.
- Choose the right size of dressing according to the wound area. If necessary, fold or cut (using sterile scissors) the dressing to adjust to wound size. **(A)**
- Apply dressing to the wound bed, without overlap onto the surrounding skin. Adopt alginate rope to cavity wounds. **(B)**
- Cover and secure alginate dressing with a secondary dressing, for example Fixomull® transparent, Hypatix® transparent or Cutimed® Siltec®. **(C) + (D)**
- Dressing change frequency will depend on patient condition, the level of exudate and the saturation of the secondary dressing. Initially it may be necessary to change the dressing every 24 hours.

**Dressing Change and Removal**

- Remove the secondary dressing.
- If the wound appears dry, saturate the dressing with sterile saline prior to removal.
- Gently remove the dressing from the wound bed and discard.
- If required, irrigate the wound site with a suitable wound cleanser prior to application of the new dressing.

**Storage**


Keep in a clean and dry place in specified temperature range and away from sunlight.

**Disposal**





Discard any unused portion of Cutimed® Alginate or used product according to local regulations, e.g. via regular household or clinical waste.

**Notice**

If any serious incident has occurred in relation to this device, it should be reported to the manufacturer and competent authority of your State.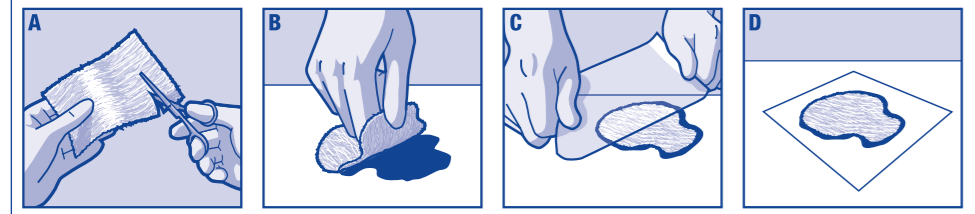

**Alginate Dressing  
Pansement d'Alginate  
Alginaatkomresse  
Alginaatverband  
Medicazione in alginato  
Apósito de alginato  
Penso de alginato  
Alginaatförband**

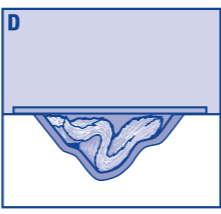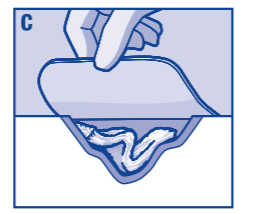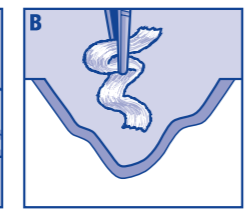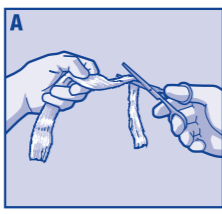**Alginate Tamponade  
Mèche d'alginate  
Alginaattamponade  
Alginaat streng  
Tampono di alginato  
Cinta de alginato  
Penso de alginato  
Alginaattamponad**

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