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ENGLISH INSTRUCTIONS FOR USE

TOXAPREVENT® SKIN POWDER for external use is a natural medical product in powder form for the treatment of open, weeping and closed skin wounds and skin diseases. It supports wound healing via the uptake of the inflammatory mediator histamine directly on the skin and/or the wound surface.

1. Indications and usage

For binding histamine and exudates in cases of injury and irritations of the skin. Particularly recommended for:

- abrasions and cuts (including cuts to the dermis)
- sore, chafed skin (also suitable for infants/young children)
- dry or weeping skin inflammations
- treatment of open leg ulcers or decubitus ulcers
- fresh burns
- bleeding injuries
- insect bites/stings (not a substitute for an anti-allergen)
- hives, urticaria

2. Dosage and duration of use

Unless otherwise instructed by a physician:

- Sprinkle powder on the appropriate part of your body as required.
- If necessary, cover the wounds with sterile dressing material.
- Use several times daily on consecutive days is recommended.
- Repeat the application only after you have cleaned the wound.
- Clean the treated site every 24 hours at the latest.
- To clean the wound, use only clean, boiled water that has cooled down to room temperature.

3. Please note

- **This is not a substitute for an anti-allergen!**
- The skin powder may be used only for the purpose specified in these instructions for use.
- Do not sprinkle the skin powder in your eyes.
- Stop using the powder if you notice any unusual effects. If necessary, consult a doctor or pharmacist.
- After using medicated creams, salves, etc., wait at least 60 minutes before applying the skin powder. In cases of doubt, consult your doctor or pharmacist about using this medical product together with other medications.
- Avoid contact with the used skin powder and the wound.
- Open the jar of powder carefully. Remove spilled powder with a damp cloth.
- After removal from the jar, the skin powder must be used only once.

4. Storage and expiration date

- Keep out of reach of children!
- Store in a dry place at a temperature below 60 °C
- Keep out of direct sunlight.
- Use by: See stamp on the edge of the label

5. Content/Ingredients

- The packet contains 18 g skin powder
- The skin powder consists of MANC® (micronized, activated, natural clinoptilolite)

6. Special instructions

The product:

- is 100% natural
- is non-allergenic
- contains no chemical additives or preservatives
- is risk-free; particularly suitable for small children
- is harmless in the event of inadvertent ingestion
- can be applied several times daily

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