# Suprasorb® P

PU-Schaumverband, selbstklebend
Pansement hydrocellulaire, adhésif
PU self-adhesive Foam Dressing
Apósito hidrocelular PU, autoadhesivo
Penso de espuma PU, adesivo
Medicazione adesiva in schiuma di poliuretano
PU schuimverband, zelfklevend
Selvklæbende sårforbinding med sårpude af PU-skum
PU skumförband, med häfta
Selvklebende sårforbinding med sårpude af PU skum
PU-haavatyyny, itseilimautuva
Polyuretanový péňový obväz, samolepici
Polyuretánový peňový obväz, samolepiacci
Óntapadó PU-habkótszer
Opatunek z pianki poliuretanowej, samoprzylepny
PU (poliuretanska) - penasta obloga, samolepilna
PU- samoljepljivi spružvasti ponovj
Αφρώδης επίδεσμος πολυουρεθάνης, αυτοκόλλητος

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## en Instructions for Use

**Product description:**

The Suprasorb® P PU foam dressing, self-adhesive, is a triple-layer wound dressing comprising a polyurethane support layer with acrylate adhesive and a polyurethane foam layer which is additionally provided with a perforated polyurethane wound contact layer. Every dressing is packed separately ready for use and sterilized with ethylene oxide. The Suprasorb® P self-adhesive PU foam dressing is disposable and must not be resterilized. The dressing is sterile provided that the packaging is unopened and intact.

Given its fluid-absorbing properties, the Suprasorb® P self-adhesive PU foam dressing is designed in such a way that it is suitable for the management of moderately exuding surface wounds. The dressing readily expands on absorbing exudate, thus providing optimal coverage for the surface of the wound.

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Adhesion to the wound is largely prevented as a moist wound healing environment is created beneath the Suprasorb® P PU foam dressing. The dressing can be changed without damaging the tissue.

The Suprasorb® P self-adhesive PU foam dressing is highly flexible and, because of its optimum adherence, is also suitable for parts of the body that are difficult to dress.

The Suprasorb® P self-adhesive PU foam dressing is also suitable to be used as a secondary dressing, e.g. in combination with Suprasorb® A.

**Indications:**

The Suprasorb® P self-adhesive PU foam dressing can be used for the treatment of all wounds provided that they are

|  |                        |
|--|------------------------|
|  | moderately exuding and |
|  | superficial            |

e.g. ulcers or bedsores.

**Contra-indications:**

- Infected wounds

**Adverse reactions:**

None known to date.

**Instructions for Use:**

**1. Wound Preparation:**

The wound should be carefully cleaned as usual and then dabbed dry. The area around the wound should be dry, grease-free and clean.

**2. Application of the Wound Dressing:**

- The size of the dressing will depend on the wound surface. The dressing should overlap the wound by at least 2 - 3 cm.
- Open the sterile packaging and remove the dressing.
- Remove the protective film.
- Smooth the dressing gently over the wound and press down at the edges.

**3. Dressing Change:**

The treating physician will stipulate how often the Suprasorb® P self-adhesive PU foam dressing should be changed. This will depend on the development of the wound condition.

- The dressing should be changed when the exudate in the expanded foam layer exceeds the size of the wound surface.
- The Suprasorb® P self-adhesive PU foam dressing can be left in place for up to max. 5 - 7 days on a carefully cleaned wound.

**Precautionary measures:**

- The Suprasorb® P self-adhesive PU foam dressing should not be used if infection is present. The prescribed compression therapy should be used in the treatment of venous leg ulcers, and pressure relief continued in the case of bedsores.
- The product may not be used in the presence of known allergies to one or several of its components.

|                      |              |          |    |
|----------------------|--------------|----------|----|
| <b>Presentation:</b> |              |          |    |
| REF                  | Size         | Pieces   |    |
| 20 416               | 7,5 x 7,5 cm | 3 x 3 in | 10 |
| 20 417               | 10 x 10 cm   | 4 x 4 in | 10 |
| 20 418               | 15 x 15 cm   | 6 x 6 in | 5  |
| 20 419               | 15 x 20 cm   | 6 x 8 in | 5  |
| 20 415 sacrum        | 18 x 20,5 cm | 7 x 8 in | 5  |

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