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Indications

- Pressure ulcers
- Leg and foot ulcers
- Traumatic wounds
- Skin tears

Suprasorb® P sensitive tracheo non-border can be used for covering skin areas around tracheal cannulae and other drains.

Contraindications
Known allergy and/or hypersensitivity to any of the product components

Application instructions
May be applied by lay persons under supervision of health care professionals. The product can be used under compression bandaging.

- Preparing the wound
 - Clean the wound in accordance with current clinical standards. Dry the surrounding skin thoroughly.
- Applying the wound dressing
 - a. The size of the dressing depends on the wound size. The wound pad should cover the dry surrounding skin by at least 1-2 cm.
 - b. Remove the first release liner and apply the adherent side to the wound.
 - c. Remove the remaining release liner(s) and smooth down on the skin. Do not stretch the dressing.

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Instructions for Suprasorb® P sensitive tracheo non-border

- Cleanse and dry the skin surrounding the tracheostomy tube in accordance with normal procedures.
- If required, the dressing can be cut to shape.
- Remove the release film from the upper side (side with the slit) and apply the adherent side of the dressing around the stoma or tracheostomy tube.
- Remove the second release film and smooth down the dressing. Press lightly.
- The dressing can be repositioned if necessary.

Precautions and warnings

- The product is for single use only.
- Do not use it if the packaging is damaged or opened.
- Do not use Suprasorb® P sensitive Silicone Foam Dressing together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.
- If redness or sensitisation occur discontinue use and consult a health care professional for appropriate treatment.
- Because of the lack of data, cautious use is advised in sensitive population groups such as infants, children, pregnant or nursing women.

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en Instructions for Use

Product description
Suprasorb® P sensitive non-border consists of a silicone wound contact layer, an absorbent PU foam and a protective backing. The triple-layer construction facilitates dynamic fluid management to provide an optimal moist wound environment, which promotes faster wound closure and may help to reduce the risk of maceration. The gentle silicone layer can be lifted and repositioned without losing its adherence. This layer can also minimise pain to patients and trauma to wounds and the surrounding skin during dressing changes. The polyurethane foam layer provides superior absorption and locks away exudate.

Product composition
Silicone, polyurethane

Intended purpose
Suprasorb® P sensitive Silicone Foam Dressing may be used to treat all wounds, provided they are

- moderately exuding and
- superficial to
- deep* :

Indications

- pressure ulcers
- leg and foot ulcers
- traumatic wounds
- skin tears

Suprasorb® P sensitive tracheo non-border can be used for covering skin areas around tracheal cannulae and other drains.

Contraindications
Known allergy and/or hypersensitivity to any of the product components

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