# Suprasorb® P sensitive border lite, multisite border lite

Silikon-Schaumverband	<span></span>
Pansement hydrocolloïdulaire siliconé	<span></span>
Silicone Foam Dressing	<span></span>
Apósito de espuma y silicona	<span></span>
Ligadura esponjosa de silicone	<span></span>
Medicazione in schiuma di silicone	<span></span>
Siliconen schuimverband	<span></span>
Silikone-skumforbinding	<span></span>
Silikonskumförband	<span></span>
Silikon-skumbandasje	<span></span>
Silikonivaahostosidos	<span></span>
Silikonový pěnový obvaz	<span></span>
Silikonový penový obvaz	<span></span>
Szilikon habkötés	<span></span>
Opatrunek z pianki z warstwą silikonu	<span></span>
Obloga iz silikonske pene	<span></span>
Povoj sa silikonom pjenom	<span></span>
Αφρώδες σιλικονούχο επίθεμα	<span></span>
Губчата повязка з сiлiкoнoм	<span></span>
Silikonlu köpük yara örtüsü	<span></span>
ضمادة السيليكون الإسفنجية	<span></span>

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### Product description

Suprasorb® P sensitive border lite consists of a silicone wound contact layer, an absorbent polyurethane foam, a non-woven distributing layer and a protective vapour permeable and waterproof backing. The four-layer construction facilitates dynamic fluid management to provide and maintain optimal moist wound environment which leads to the promotion of faster wound closure and may help to reduce the risk of maceration. The gentle silicone layer can be lifted and repositioned without losing its adherence. Also, this layer can minimise pain to patients and trauma to wounds and the surrounding skin during dressing changes. The PU foam layer provides superior absorption and locks exudates away.

### Product composition

Silicone, polyurethane, polyacrylate, viscosse and polyester

<span></span>	<span>Intended purpose</span>
<span></span>	<span>Suprasorb® P sensitive Silicone Foam Dressing may be used to treat all wounds, provided they are</span>

<span></span>	<span>superficial to</span>
<span></span>	<span>deep*.</span>

<span></span>	<span>Indications</span>
<span></span>	<span>• Pressure ulcers</span>
<span></span>	<span>• Leg and foot ulcers</span>
<span></span>	<span>• Traumatic wounds</span>
<span></span>	<span>• Surgical wounds</span>
<span></span>	<span>• Skin tears</span>

<span></span>	<span>Kontraindications</span>
<span></span>	<span>Known allergy and/or hypersensitivity to any of the product components</span>

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- Remove the first release liner and apply the adherent side to the wound.
- Remove the remaining release liner(s) and smooth down the border on the skin. Do not stretch the dressing.
  - \*Where necessary, for deep wounds apply a rope as primary dressing (e.g. Suprasorb® A, Suprasorb® A+Ag, Suprasorb® Liqueal).
- Dressing change
  - The dressing can remain on the wound for several days, dependent on wound condition and the level of exudate.

<span></span>	<span>Precautions and warnings</span>
<span></span>	<span>• The product is for single use only.</span>
<span></span>	<span>• Do not use it if the packaging is damaged or opened.</span>
<span></span>	<span>• Do not use Suprasorb® P sensitive Silicone Foam Dressing together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.</span>
<span></span>	<span>• If reddening or sensitisation occur discontinue use and consult a health care professional for appropriate treatment.</span>
<span></span>	<span>• Because of the lack of data, cautious use is advised in sensitive population groups such as infants, children, pregnant or nursing women.</span>

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<span></span>	<span>Product description</span>
<span></span>	<span>Suprasorb® P sensitive border lite consists of a silicone wound contact layer, an absorbent polyurethane foam, a non-woven distributing layer and a protective vapour permeable and waterproof backing. The four-layer construction facilitates dynamic fluid management to provide and maintain optimal moist wound environment which leads to the promotion of faster wound closure and may help to reduce the risk of maceration. The gentle silicone layer can be lifted and repositioned without losing its adherence. Also, this layer can minimise pain to patients and trauma to wounds and the surrounding skin during dressing changes. The PU foam layer provides superior absorption and locks exudates away.</span>

<span></span>	<span>Product composition</span>
<span></span>	<span>Silicone, polyurethane, polyacrylate, viscosse and polyester</span>

<span></span>	<span>Intended purpose</span>
<span></span>	<span>Suprasorb® P sensitive Silicone Foam Dressing may be used to treat all wounds, provided they are</span>

<span></span>	<span>superficial to</span>
<span></span>	<span>deep*.</span>

<span></span>	<span>Indications</span>
<span></span>	<span>• Pressure ulcers</span>
<span></span>	<span>• Leg and foot ulcers</span>
<span></span>	<span>• Traumatic wounds</span>
<span></span>	<span>• Surgical wounds</span>
<span></span>	<span>• Skin tears</span>

<span></span>	<span>Kontraindications</span>
<span></span>	<span>Known allergy and/or hypersensitivity to any of the product components</span>

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