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Single-use sterile medical device

DESCRIPTION

UrgoTul Duo is a lite absorbent dressing designed and developed by Laboratoires URGO. UrgoTul Duo is made of a polyester textile mesh coated with TLC matrix (Technologie Lipido-Colloïde) and a thin absorbent pad. The TLC matrix is composed of petroleum jelly, liquid paraffin, sodium carboxymethylcellulose (CMC), and polymers. Each UrgoTul Duo dressing has protective tabs and is packaged in an individual pouch that maintains its sterile condition.

INDICATIONS

UrgoTul Duo is indicated for covering and treating skin wounds. UrgoTul Duo is suitable for application to wounds with difficult skin location (for example, heel or finger). UrgoTul Duo must be secured with a dressing securement system (for example, medical tape, bandage, or tubular mesh).

Target patients

Adults, children and babies. In the absence of specific data, the use of UrgoTul Duo in pregnant or breastfeeding women is not indicated.

PROPERTIES AND CLINICAL BENEFITS

Treatment with UrgoTul Duo promotes the healing of skin wounds. In contact with wound exudate (fluid in the wound), the TLC matrix of UrgoTul Duo gelifies, which helps to maintain a moist environment necessary for wound healing.

UrgoTul Duo :

- Is easy to apply.
- Is flexible and conformable (adapts to the body area to which it is applied).
- Is suitable to also cover the perilesional skin (skin surrounding the wound) even when it is altered.
- Is painless and atraumatic to the wound bed and the perilesional skin, when removed.

CONTRA-INDICATIONS

- Mucous membrane wounds.
- Known hypersensitivity of the patient to one of the TLC dressings or to carboxymethylcellulose (CMC).

POTENTIAL ADVERSE SIDE EFFECTS

Consult a healthcare professional in the event of skin reactions (for example, redness, inflammation, especially on the skin around the wound), eczema or other allergic-type reactions (for example, itching).

Any signs or symptoms occurring during treatment with UrgoTul Duo should be reported to a healthcare professional. Any serious incident that occurs in relation to UrgoTul Duo should be reported to the manufacturer (Laboratoires URGO) and the national competent authority.

PRECAUTIONS FOR USE

In the event of signs or symptoms of infection before or after starting treatment with UrgoTul Duo, such as erythema (redness of the skin around the wound), oedema (swelling), heat or pain in the wound area, fever, enlargement of the wound, odour from the wound, or profuse exudate (that is, a large amount of fluid in the wound), consult a healthcare professional for appropriate treatment.

WARNINGS

- Do not use a previously used dressing.
- Discard all unused cut pieces of UrgoTul Duo, as the sterile condition is not maintained anymore.
- Do not re-sterilize.
- Do not use beyond expiration date.
- Keep the pouches of UrgoTul Duo in their box until use.

The use of a dressing that is in non-sterile condition, re-sterilized, or beyond expiration date, does not guarantee the performances of UrgoTul Duo and exposes the patient to the risk of infection.

DIRECTIONS FOR USE

UrgoTul Duo can be applied by a healthcare professional, or by the patient or a relative after training with a healthcare professional.

Before use

1. Respect the rules of hygiene and asepsis: disinfect your hands or wash them with soap and water; or wear sterile gloves.
2. Prepare the wound according to the usual wound management protocol.
3. Gently dry the skin around the wound.
4. Provide an appropriate securement system (for example, medical tape, bandage, or tubular mesh).

To apply UrgoTul Duo

5. Adapt the size of the dressing to ensure it exceeds the surface of the wound. If needed, UrgoTul Duo can be cut with sterile scissors.
6. Check that the pouch that maintains the sterile condition of UrgoTul Duo has not been damaged or unintentionally opened before use. If this is the case, use another one.
7. Open the pouch on its full length.
8. Take UrgoTul Duo according to hygiene and asepsis rules.
9. Remove the protective tabs and apply the dressing to the wound.
10. Apply the securement system previously prepared.

If any compression therapy is prescribed, apply the compression system.

When appropriate, set up the foot offloading system.

Replacement frequency and treatment duration

The replacement frequency is left to the discretion of the healthcare professional. However, it is recommended to replace UrgoTul Duo every **1 to 4 days** and then to adapt the replacement frequency according to the clinical assessment of the wound and the level of exudate (that is, the amount of fluid in the wound) **without exceeding 7 days of use per dressing**.

It is recommended to continue treatment with UrgoTul Duo until complete wound healing.

The time to complete healing depends on the characteristics of the wound (for example, its size, duration, and recurrence) and of the patient (for example, age, mobility, and comorbidities).

To discard the dressing

UrgoTul Duo as well as the disposable articles used to prepare the wound or apply the dressing, should be disposed of safely to avoid infection and microbial hazards.

In domestic use, put all these consumables in disposable packaging, and discard them all with household waste. In hospital, dispose of according to the procedures in force.

	MEDICAL DEVICE		CAUTION. READ THE PRECAUTIONS FOR USE / WARNINGS / CONTRAINDICATIONS
	STERILISED USING RADIATION		UNIQUE DEVICE IDENTIFIER
	SINGLE STERILE BARRIER SYSTEM (POUCH MAINTAINING STERILE CONDITION)		CATALOGUE NUMBER
	DO NOT USE IF PACKAGE IS DAMAGED AND CONSULT INSTRUCTIONS FOR USE		BATCH CODE
	DO NOT RE-USE		USE-BY DATE
	DO NOT RE-STERILIZE		MANUFACTURER RESPONSIBLE FOR MARKETING
	CONSULT INSTRUCTIONS FOR USE		MANUFACTURING DATE

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