Merula® #XL
menstrual cup

Gebrauchsanleitung
Manual
Mode d'emploi
Használati utasítás
Manuale
Gebruikershandleiding
Instrukcja obsługi
Manual de Instruções
инструкция
Návod

heavy flow

Die moderne Art der Monatshygiene ★ The modern kind of
sanitary products ★ La forma moderna de productos sanitarios
★ L'hygiène menstruelle moderne ★ Modern menstruációs
termék ★ Il moderno nei prodotti sanitari ★ De moderne man-
ier van maandelijke hygiëne ★ Nowoczesny produkt higieny
osobistej ★ O tipo mais moderno de produtos de higiene ★
Современное средство гигиены ★ Menstruační kalíšek to je
moderní způsob menstruace

VEGAN

VEGANO

VÉGÉTALIEN

VEGÁN

VEGANA

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IMPORTANT INFORMATION

Do not use the Merula Cup XL until you have completely read and understood the instruction manual. Store this instruction manual. This instruction manual is not a substitute for medical advice. If you have any doubts before, during or after use of the Merula Cup XL, please consult your doctor. The Merula Cup XL is a sanitary product and no returns and/or exchanges are allowed. All sales are final. We reserve the right to make changes to the instruction manual without separate notice. You can always download the latest version of the Merula Cup XL instruction manual on our website, www.merula-cup.de/en.

Advantages of the Merula Cup XL

Ecological

- ★ One cup can be used for several years
- ★ Produces far less trash than disposable products

Economical

- ★ There is no need to frequently buy expensive supplies
- ★ After a few months, the small investment in a Merula Cup XL does pay

Comfortable

- ★ Big capacity
- ★ Does not absorb water during swimming or in the bath tub
- ★ No thread dangling from the body (invisible)

What are the particularities of the Merula Cup XL?

- ★ It has a very large capacity (up to 50 ml / 1.691 fl oz) with a diameter of 46 mm / 1.811 in.. The Merula Cup XL adapts perfectly to your body and is an ideal companion for very heavy bleeding.
- ★ From weak to strong pelvic floor muscles: The Merula Cup XL stays where it belongs. As it can be worn in a folded up state and does not have to open completely, even strong pelvic floor muscles do not make it lose its shape.
- ★ The Merula Cup XL offers maximum capacity at a cup height of only 50 mm / 1.969 in., which is roughly equal to 2 average, conventional menstrual cups or

2.5 large tampons. As such, the Merula Cup XL fits women with a medium to high cervix. Thanks to its two rung ladder stem, it can be ideally adapted to your personal needs.

★ The Merula Cup XL is suitable for a medium to high cervix. Should your cervix be exceptionally low or if you feel your vagina is exceptionally "tight", then the Merula Cup XL may be too large for you. In this case, we recommend the Merula Cup "one size" with a capacity of 38 ml / 1.285 fl oz.

Insertion & Removal

Punchdown-Fold

Push the rim of the Merula Cup at one point in direction to the inner bottom and fold the whole cup. As you do so, the cup becomes smaller and it is easier to insert.

The cup can be moistened with a little bit of water. You can also use *Merula Lube* (or another water-based lubricant), which makes the cup extremely easy to insert. Insert the folded cup and make sure that the rim opens again when inside. The bottom of the cup can stay collapsed. If the cup leaks, try to correct its position. The cup does not have to be inserted as deep as a tampon.

For removal, press the cup slightly downwards and hold the stem. Move the cup downwards, with slow zigzag movements, until you can reach the bottom. Press the bottom in order to remove the light suction. Now, you can remove the cup completely and clean it.

Should insertion or removal not function properly right away, please practice fully relaxed for a few cycles and, if necessary, take some breaks. You will find further advice on our website: www.merula-cup.de/en

Cleaning

Boil the cup before the first use and after every menstruation. Use water with a pinch of vinegar in a pot or in a microwave and boil the cup for at least 5 minutes. The cup can be used up to 8 hours at once, but can also be removed and cleaned more frequently.

After 8 hours, empty the contents of the cup into the toilet and rinse the cup with water. In order to prevent staining, rinse the cup with cold water first. Then, before reinserting the cup, clean it with some warm water and *Merula Spray* (designed to gently clean and disinfect your Merula Cup XL) or another disinfectant that is suitable for a sex toy. If you're out and about and have to empty your cup in a bathroom without a private sink, we recommend using our handy *Merula Wipes* for cleaning. As a last resort, you can also thoroughly rub and wash out your cup with some bottled water. But be sure to clean it with *Merula Spray*, *Merula Wipes*, or a comparable disinfectant as soon as possible afterwards, for it must be cleaned with one of these products at least once every 8 hours.

Keep the cup always in a container that is permeable to air, such as the bag that is delivered with Merula. Well maintained, a single cup can be used for several years. If the silicone begins to deteriorate, throw it away and buy a new one. Slight staining due to frequent use is normal.

Adaptation

First, try your Merula Cup XL. After that, you can adapt the stem to your individual length.

Remove the parts that are highlighted with nail scissors to adapt the stem to your required length. The cup should disappear completely inside your body, but still be easily reachable with at least one finger.

Safety instructions

- ★ Always wash your hands thoroughly before and after using the cup.
- ★ Always loosen the light suction before removing the cup.
- ★ The Merula Cup XL should be removed before sexual intercourse and is neither suitable for contraception nor does it protect against sexually-transmitted diseases.
- ★ Clean the cup at least every 8 hours and boil it for a minimum of five minutes or more at least once after the end of the menstruation.
- ★ If you wear an intrauterine device (coil/chain/ball), consult your doctor beforehand on using a menstrual cup.
- ★ Dispose of the cup properly in the trash and not in the toilet.
- ★ Always store your cup out of the reach of children and animals.
- ★ If you have any doubts or experience pain while using the cup, remove it immediately and consult with a doctor.

★ Menstrual Toxic Shock Syndrome (TSS)

TSS is a rare but dangerous disease, which can occur with men, women, and children. At the moment, there have not been many known cases to be connected with a menstrual cup, but still: extreme cases of TSS can lead to death.

The following symptoms, which can develop very quickly and appear to only be influenza in the beginning:

Sudden high fever ($>39^{\circ}\text{C}$ / $>102^{\circ}\text{F}$), nausea, diarrhea, sunburn-like rash, sore throat, vertigo, and/or fainting.

If you experience one or more of these symptoms during your menstruation, remove the cup and consult with a doctor immediately. Mention TSS and that you are menstruating. This way, it is possible to diagnose the disease early and treat it effectively. If you are already affected by TSS, consult with your doctor before using cups or tampons.

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