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Alcohol-free, ready-to-use disinfection for wipe-resistant surfaces on medical devices

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Ready-to-use, alcohol- and aldehyde-free disinfectant solution for the disinfection of wipe-resistant surfaces of non-invasive medical devices and for surfaces in medical environments, food-processing areas, public facilities, and industrial settings. Not intended for use on instruments or on semi-critical and critical medical devices.

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**Important Note:** Non-observance of the information, especially in view of use, compatibility and durability may lead to damage to persons and / or material. Meditrade does not take any liability in case of improper use. This is a single use product – changes in material / infection risk cannot be excluded in case of multiple use. All information included correspond with the last document amendment date. Errors, literal mistakes and changes expected. In case of doubt please seek expert advice. Please find any up-dated product information on our homepage [www.meditrade.de](http://www.meditrade.de).

Remove visible soil before disinfection. Thoroughly wet surfaces and, if necessary, apply using a clean, single-use wipe. Observe the specified contact times. Keep surfaces wet for the entire contact time. For surfaces and equipment that come into contact with food, rinse thoroughly with potable water after the contact time has elapsed. Test material compatibility on an inconspicuous area before use.

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Wirkstoffe in 100 g / **Active substances in 100 g:**  
0,1g 2-Phenoxyethanol  
0,35g Alkyl(C12-16) dimethylbenzylammoniumchlorid  
0,25g N-(3-Aminopropyl)-N-dodecylpropan-1,3-diamin

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3 years (see label). Shelf life after breaking seal: 12 months. Keep containers closed tightly. Warehousing and storage under standard storage conditions (protect from moisture, direct sunlight, heat and frost).

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**Dual use:**

- Medical device class IIa according to directive 93/42/EWG
- Biocide product: 120176

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Harmful to aquatic life with long-lasting effects. Avoid release to the environment. Dispose of contents/container in accordance with local and national regulations. Keep out of reach of children.