Antimicrobial, odour adsorbing primary wound dressing.

Antimicrobial and odour adsorbing primary dressing. For single use only. Store in a dry place (<25°C / <77°F). Avoid exposure to temperatures above 50°C/122°F.

Description

Zorflex LA dressing consists of a pure activated carbon cloth with a low adherent layer applied to both sides. It is a comfortable dressing that maintains contact with the wound surface. The Zorflex LA dressing is black to grey in colour, variation in colour is possible, however this does not affect the performance. Zorflex LA provides an effective anti-microbial barrier to microbial contamination protecting the wound from invasive micro-organisms which may prevent the wound from healing. Zorflex LA is also effective against micro-organisms already present in the wound. The anti-microbial barrier properties of the Zorflex®-la remain effective for a minimum of 7 days. Zorflex® has been shown to reduce odour, pain and exudation. Zorflex® is conductive. Zorflex LA can be used either dry or moistened with sterilised water.

Indications

Zorflex LA is indicated as an antimicrobial low adherent dressing over discharging, partial and full thickness wounds such as: cavities and fistulas, trauma, surgical sites, fungating carcinomas, pressure ulcers, venous ulcers, diabetic ulcers, burns, donor and recipient graft sites.

Contra-indications

None.


Precautions

Use sterile water to cleanse, irrigate or moisten the wound. The use of oil based products, topical ointments or saline will reduce the effectiveness of Zorflex LA. Do not reuse. If reused the performance of the product may deteriorate and cross contamination may occur.

Instructions for use

- Cleanse the wound with sterile water.
- Cut the dressing to shape, covering the periwound if maceration is evident. Do not tear.
- Apply Zorflex LA either way up to the wound surface.
- Secure the Zorflex LA dressing in place using an appropriate secondary dressing or compression therapy. Alternatively secure with a dressing retention sheet, tape or bandage. In the case of highly exudating wound an absorbent secondary dressing is appropriate. Zorflex LA allows transfer of the exudate into the secondary dressing which should be replaced as necessary without disturbing the Zorflex LA or the wound bed.
- Should the dressing dry or adhere to the wound, moisten or soak the dressing with sterile water to assist removal and avoid disruption of the healing wound.
- Zorflex LA can be left on the wound for up to 7 days, or as clinically indicated. Any threads or fibres which detach from the dressing can be easily seen and removed.
- Device should only be used as specified.
- Device should be used under relevant supervision by healthcare professionals.

Instructions for use			
Product Availability			
Code	Size	Availability	
ZLA10	10.5cm x 10.5cm (4 1/4" x 4 1/4")	Pouched only	
ZLA15	15cm x 25cm (6" x 10")	Pouched only	
ZLA20	20cm x 10cm (8" x 4")	Pouched only	
Manufacturer			
Chemvion Carbon Limited, Cloth Division, Rainton Bridge Industrial Estate Houghton-le-Spring, Tyne & Wear, DH4 5PP, GB			
Zorflex® trade mark of Chemvion Carbon.			

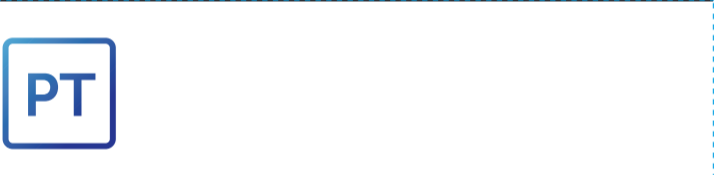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