Produktbeschreibung

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EN Instructions for use

Product description

PermaFoam Classic is a hydrophilic polymer with a three-dimensional foam structure which can effectively absorb wound exudate and maintain a moist wound environment.

PermaFoam Classic Border also has an adhesive border coated with a polyacrylate adhesive.

Composition

- PU foam
- Polycrylate
- PU film

Indications

PermaFoam Classic is a sterile, non-medicated single-use polyurethane foam dressing indicated for medium to highly exuding, acute and chronic wounds, such as venous ulcers (leg ulcers), pressure ulcers II-IV, diabetic foot ulcers, donor sites, abrasions, incisions.

Mode of application

1. Clean the wound with a physiological solution or as instructed by the clinician and dry the surrounding skin gently.

2. Choose a suitable dressing size for the wound making sure that the central pad exceeds the edges of the wound by approx. 1-2 cm.

3. When using the non-adhesive version of PermaFoam Classic: Place PermaFoam Classic with the white side on the wound so that the beige outer layer is facing away from the wound. Secure the dressing with an additional fixation.

4. When using PermaFoam Classic Border: Remove the release liners, apply the white side of the foam pad to the wound and press down the adhesive edges firmly.

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FR – Lab. PAUL HARTMANN S.A.r.l. · Châtenois – 67607 Sélestat CEDEX
IT – PAUL HARTMANN S.p.A. · 37139 Verona