beurer

HK 145 Cosy Weight Green Planet

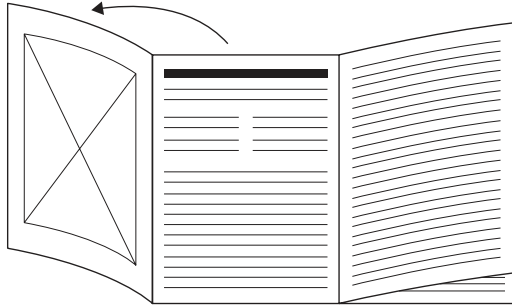
DE Heizkissen Gebrauchsanweisung	4
EN Heating pad Instructions for use.....	9
FR Coussin chauffant Mode d'emploi.....	14
ES Almohadilla eléctrica Instrucciones de uso.....	19
IT Termofofo Istruzioni per l'uso	24
TR Isı yastığı Kullanım kılavuzu.....	29
RU Электрическая грелка Инструкция по применению	34
PL Poduszka elektryczna Instrukcja obsługi	39
NL Warmtekussen Gebruiksaanwijzing	44
DA Varmepude Betjeningsvejledning.....	49
SV Värmekudde Bruksanvisning.....	54
NO Varmepute Bruksanvisning.....	59
FI Lämpötyyny Käyttöohje.....	64


**OEKO
TEX®** STANDARD
100
Special Articles
24.HDE.72388
Hohenstein HTTI

www.oeko-tex.com

CE

- DE** Klappen Sie vor dem Lesen der Gebrauchsanweisung die Seite 3 aus.
- EN** Unfold page 3 before reading the instructions for use.
- FR** Dépliez la page 3 avant de lire le mode d'emploi.
- ES** Despliegue la página 3 antes de leer las instrucciones de uso.
- IT** Prima di leggere le istruzioni per l'uso aprire la pagina 3.
- TR** Kullanım kılavuzunu okumadan önce 3. sayfayı açın.
- RU** Перед чтением инструкции по применению разложите страницу 3.
- PL** Przed przeczytaniem instrukcji obsługi otworzyć stronę 3.
- NL** Vouw pagina 3 uit om de gebruiksaanwijzing te kunnen lezen.
- DA** Fold side 3 ud, før du læser betjeningsvejledningen.
- SV** Vik ut sid. 3 innan du läser bruksanvisningen.
- NO** Åpne side 3 før du leser bruksanvisningen.
- FI** Käännä sivu 3 auki ennen käyttöohjeen lukemista.

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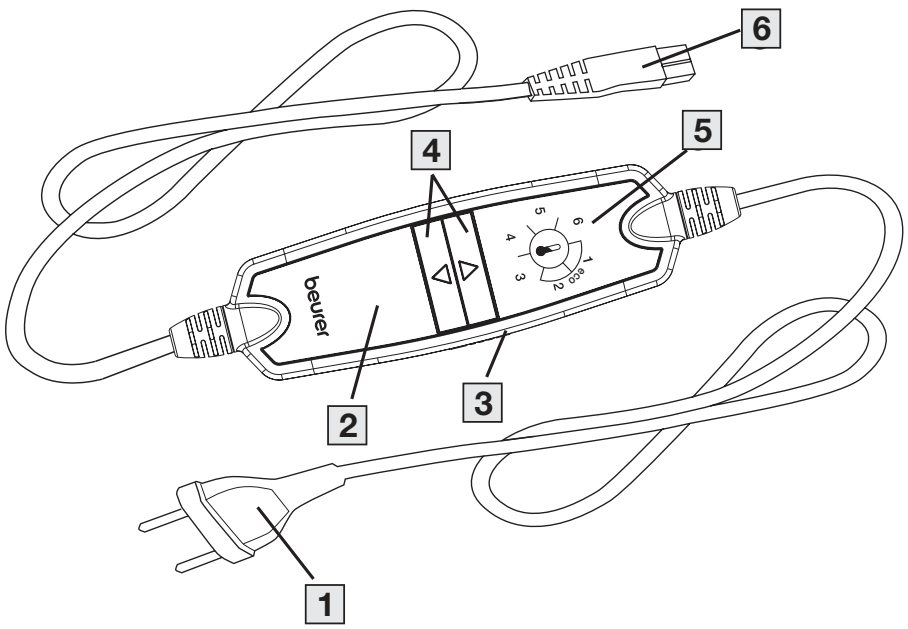 **Read these instructions for use carefully. Observe the warnings and safety notes. Keep these instructions for use for future reference. Make the instructions for use accessible to other users. If the device is passed on, provide the instructions for use to the next user as well.**

TABLE OF CONTENTS

1. Included in delivery	9	6.4 Setting the temperature	12
2. Signs and symbols	9	6.5 Automatic switch-off	12
3. Important instructions Retain for future use	10	6.6 Switching off	12
4. Intended use	11	7. Cleaning and maintenance	12
5. Device description	11	8. What if there are problems?	12
6. Operation	11	9. Storage	12
6.1 Safety	11	10. Disposal	13
6.2 Initial use	12	11. Technical Specifications	13
6.3 Switching on	12	12. Guarantee / Service	13

1. INCLUDED IN DELIVERY

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address. Remove the hang tag with information on Beurer Green Planet before initial use.

- 1 Heat pad
- 1 Control
- 1 Instructions for use

2. SIGNS AND SYMBOLS

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

⚠ DANGER
Indicates an imminent danger. If it is not avoided, death or serious injury will occur.
⚠ WARNING
Indicates a potentially impending danger. If it is not avoided, there is a risk of death or serious injury.
⚠ CAUTION
Indicates a potentially imminent hazard. If it is not avoided, slight or minor injuries may result.
NOTICE
Indicates a potentially harmful situation. If it is not avoided, the device or something in its vicinity may be damaged.
 Product information Note on important information
 Read the instructions

	The electronic device must not be disposed of with household waste
	Manufacturer
	CE labelling This product satisfies the requirements of the applicable European and national directives.
	Marking to identify the packaging material. A = Material code, B = Material number: 1-7 = Plastics, 20-22 = Paper and cardboard
	Separate the product and packaging elements and dispose of them in accordance with local regulations.
	Protection class II device
	The products demonstrably meet the requirements of the Technical Regulations of the EAEU.
	Must not be used by very young children (0-3 years old).
	United Kingdom Conformity Assessed Mark
	Importer
	Wash at a maximum temperature of 30 °C Very gentle wash
	Do not bleach
	Do not dry clean
	Do not insert pins!
	Do not iron
	Do not tumble dry

	Do not use when folded or bunched up!
	GRS – standard for independent certification of recycled material
	The textiles used for this device meet the stringent human ecological requirements of Oeko Tex Standard 100, as verified by Hohenstein Research Institute.
	Smartphone link to background information about Beurer Green Planet products and sustainability

3. IMPORTANT INSTRUCTIONS RETAIN FOR FUTURE USE

▲ WARNING

- Non-observance of the following notes may result in personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information is not only intended to protect your health and the health of others, it should also protect the product. For this reason, pay attention to these safety notes and include these instructions when handing over the product to others.
- This heat pad must not be used by persons who are not sensitive to heat or by other vulnerable persons who may not be able to react to overheating (e.g. diabetics, people with skin alterations due to illness or scarred tissue in the application area, after taking pain relief medication or alcohol).
- This heat pad must not be used by very young children (0-3 years old) as they are unable to respond to overheating.
- The heat pad can be used by children older than 3 and younger than 8 years of age provided they are supervised. For this, the control must

always be set to the minimum temperature.

- This heat pad may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the heat pad safely, and are fully aware of the consequent risks of use.
- Children must not play with the heat pad.
- Cleaning and user maintenance must not be performed by children unless supervised.
- This heat pad is not designed for use in hospitals.
- This heat pad is only intended for domestic/private use, not for commercial use.
- Do not insert pins.
- Do not use folded or rucked.
- Do not use if wet.
- This heat pad may only be used in conjunction with the control specified on the label.
- This heat pad must only be connected to the mains voltage that is specified on the label.
- If the heat pad is being used over several hours, we recommend that you set the lowest temperature setting on the control to avoid the body overheating and thereby any potential burns to the skin or heat stroke.
- The electrical and magnetic fields emitted by this heat pad may interfere with the function of a pacemaker. However, they are still well below the limits: electrical field strength: max. 5000 V/m, magnetic field strength:

max. 80 A/m, magnetic flux density: max. 0.1 millitesla. Please therefore consult your doctor and the manufacturer of your pacemaker before using this heat pad.

- Do not pull, twist or make sharp bends in the cables.
- If the cable and control of the heat pad are not positioned properly, there may be a risk of becoming entangled in, being strangled by, tripping over, or stepping on the cable and control. The user must ensure that excess lengths of cable, and cables in general, are safely routed.
- Please check this heat pad frequently for signs of wear and tear or damage. If any such signs are evident, if the heat pad has been used incorrectly or if it no longer heats up, it must be checked by the manufacturer before being switched on again.
- If glass beads are released due to damage to the product, stop using the product. Never swallow the glass beads. If glass beads are swallowed, consult a doctor immediately.
- Under no circumstances should you open or repair the heat pad (including the accessories) yourself because faultless functionality can no longer be guaranteed thereafter. Failure to observe this will invalidate the guarantee.
- If the mains connection cable of this heat pad is damaged, it must be disposed of. If it cannot be removed, the heat pad must be disposed of.
- Do not expose control and cables to direct sunlight.
- When this heat pad is switched on:
 - Do not place any sharp objects on it

-Do not place any heat sources, such as hot water bottles, heat pads or similar, on it

- The electronic components in the control warm up when the heat pad is in use. For this reason, the control must never be covered or placed on the heat pad when it is in use.
- Allow the heat pad to cool down before storing it away. Otherwise the heat pad may get damaged.
- To avoid any creasing in the heat pad, do not place any objects on top of it while it is being stored. Otherwise the heat pad may get damaged.
- It is essential to observe the information relating to the following chapters: Operation, Cleaning and maintenance, Storage.
- If you should have any questions about using our devices, please contact our Customer Services department.

4. INTENDED USE

⚠ CAUTION

This heat pad is only designed to warm up the human body.

5. DEVICE DESCRIPTION

The related illustrations are on page 3.

- 1 Power plug
- 2 Control
- 3 Sliding switch (ON = 1 / OFF = 0)
- 4 Buttons for setting temperature
- 5 Illuminated display for temperature settings
- 6 Plug-in coupling

6. OPERATION

6.1 Safety

NOTICE

- The heat pad is fitted with a SAFETY SYSTEM. This sensor technology provides protection against overheating across the entire surface of the heat pad with an automatic switch-off in the event of a fault. If the SAFETY SYSTEM has switched off

the heat pad, the temperature settings are no longer illuminated when switched on.

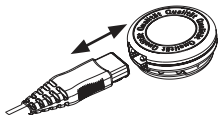
- Please note that for safety reasons, the heat pad can no longer be operated after a fault has occurred and must be sent to the specified service address.
- Do not connect the defective heat pad with another control of the same type. This would trigger a permanent switch-off via the control's safety system.

6.2 Initial use

NOTICE

Ensure that the heat pad will not bunch up or become folded during use.

1. To operate the heat pad connect the control to the heat pad by plugging in the connector.
2. Then plug in the power plug into the mains outlet.

6.3 Switching on

Push the sliding switch **3** on the right side of the control to the setting "I" (ON) – see image of control. When the switch is on, the temperature settings display is illuminated.

-  Gently shake the pad regularly to ensure the filling is evenly distributed.

6.4 Setting the temperature

To increase the temperature, press the Δ button **4**. To reduce the temperature, press the ∇ button **4**.

Level 1: minimum heat

Level 2-5: individual heat setting

Level 6: maximum heat

-  The fastest way to warm up the heat pad is to initially set the highest temperature setting.

-  This heat pad has a fast heating function, which allows the pad to warm up faster in the first 10 minutes.

6.5 Automatic switch-off

This heat pad is equipped with an automatic switch-off function. This turns off the heat supply approx. 90 minutes after the initial use of the heat pad. A part of the displayed temperature settings on the control then begins to flash. So that the heat pad can be switched back on, the side sliding switch **3** must first be set to setting "0" (OFF). After about 5 seconds it is possible to switch it on again.

WARNING

If the heat pad is being used over several hours, we recommend that you set the lowest temperature setting on the control to avoid overheating the heated body part, which may lead to burns to the skin.

6.6 Switching off

To switch the heat pad off, set the sliding switch **3** on the side of the control to setting "0" (OFF). The temperature settings display is no longer illuminated.

-  If the heat pad is not in use, switch the side sliding switch **3** for ON/OFF to setting "0" (OFF) and unplug the power plug from the socket. Then disconnect the control from the heat pad by unplugging the plug-in coupling.

7. CLEANING AND MAINTENANCE

WARNING

Before cleaning, always remove the power plug from the socket first. Then disconnect the control from the heat pad by unplugging the plug-in coupling. Otherwise there is a risk of electric shock.

NOTICE

The control must never come into contact with water or other liquids, as this may cause damage.

- To clean the control, use a dry, lint-free cloth. Do not use any chemical or abrasive cleaning agents.
- Small marks on the heat pad can be removed with a damp cloth and if necessary, with a little liquid detergent for delicate laundry.

NOTICE

Please note that the heat pad may not be chemically cleaned, wrung out, tumble dried, put through a mangle or ironed. Otherwise the heat pad may be damaged.

- This heat pad is machine-washable.
- Set the washing machine to a particularly gentle wash cycle at 30 °C (wool cycle). Use a delicate laundry detergent and measure it out according to the manufacturer's instructions.

NOTICE

Please note that frequent washing of the heat pad has a negative effect on the product. The heat pad should therefore be washed in a washing machine a maximum of 10 times during its life.

- Immediately after washing, reshape the heat pad to its original dimensions while it is still damp and spread it out flat on a clothes horse to dry.

NOTICE

- Do not use pegs or similar items to attach the heat pad to the clothes horse. Otherwise the heat pad may be damaged.
- Do not reconnect the control to the heat pad until the plug-in connection and the heat pad are completely dry. Otherwise the heat pad may be damaged.

WARNING

Never switch the heat pad on to dry it! Otherwise there is a risk of electric shock.

8. WHAT IF THERE ARE PROBLEMS?

Problem	The temperature settings are not illuminated - while the control is properly connected to the heat pad - the power plug is connected to a working socket - the side sliding switch on the control is set to setting "I" (ON)
Cause	The safety system has switched the heat pad off permanently.
Solution	Send the heat pad and the control for servicing.

9. STORAGE

If you do not plan to use the heat pad for a long period of time, we recommend that you store it in the original packaging. For this purpose, disconnect the control from the heat pad by unplugging the plug-in coupling.

10. DISPOSAL

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal..

11. TECHNICAL SPECIFICATIONS

See the rating label on the heat pad.

12. GUARANTEE / SERVICE

Further information on the guarantee and guarantee conditions can be found in the guarantee leaflet supplied.

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