AquaNox

Produktbeschreibung

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AquaNox

Product description

AquaNox is a wound irrigation solution for cleansing, rinsing and moisturizing of wounds. For the management of minor skin abrasions, minor lacerations, minor irritations, minor cuts, minor burns and intact skin – helps prevent biofilm.

1. Identification, composition and use

AquaNox is an easy to use spray liquid formulation which is an electrolyzed oxidizing water solution known as hypochlorous acid that is dermatologically tested. For:

- Cleansing and debriding of the wound and preventing infections.
- For the management of minor skin abrasions, minor lacerations, minor irritations, minor cuts, minor burns and intact skin.
- Wound irrigation solution.
- To irrigate and moisten.
- For sore irritated skin.
- Helps with the prevention of biofilm
- Can be used for moistening the wound dressing and moisturising the wound itself as well as for loosening crusted dressings.
- Flushing of irritants associated with wound malodour.

1.1 Mechanism of action

AquaNox is a wound irrigation solution and is an electrochemically modified preserved water solution using hypochlorous acid. Its key mode of action is to modify the wound environment by mechanical removal of foreign bodies that may cause skin irritation and infection and to provide an optimal environment with regards to moisture, humidity, oxygen and pH to assist in the healing process. Alcohol free, suitable from birth and no need to rinse.

1.2 Components

Each bottle AquaNox contains:

- 0.016 % hypochlorous acid
- Excipients: ionised water, sodium chloride

2. Package

AquaNox is available in the following presentations:

60 ml, 150 ml, 250 ml bottles with finger spray or nozzle or 500 ml, 1000 ml bottle with trigger spray or screw cap.

3. Dosage and method for use

Apply AquaNox to the affected area to rinse the wound, thereby removing any foreign bodies. Apply as required for optimal modification of the wound environment.

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4. Side effects

In rare cases a minor sting sensation may occur.

5. Interactions

No known interactions.

6. Storage

Store AquaNox at room temperature, maximum 25°C, away from direct heat and sunlight. Do not store below 4°C. Do not transfer into another container. Recap after each application. Keep out of reach of children.

7. Warnings and precautions

Do not use AquaNox in case of known hypersensitivity or intolerance to any constituent, contact your healthcare professional for any queries.
Do not ingest AquaNox.
AquaNox is suitable from birth.
Medical Device is Alcohol Free.
For best effect allow spray to dry naturally.
For Topical Use Only.
Do not use for more than 30 days.
Concomitant use of product with other medicaments should only be carried out with healthcare professional advice.

8. Expiration Date

Do not use after expiration date printed on the packaging.
The expiry date refers to the AquaNox in its intact container when stored.

9. Disposal

Expired device must be managed according to the waste management regulations of Local Entity Manager. Do not dispose AquaNox or its packaging into the environment.