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EN Instructions for use

Please read and follow guidance given in instructions for use of Safety Huber Needles for Single Use

Product structure, type and model

Safety Huber Needles are assembled by needle component, tubing, tubing insert, flow clip, female conical fitting, lock cover.

Specifications can be identified by the color coding of needle handle, needle hub and/or protective cap: mm

SIZE	18G (1,3mm) x 15mm	18G (1,3mm) x 19mm
REF	09/01/18G/15	09/01/18G/19
COLOR CODE	PINK	PINK
SIZE	18G (1,3mm) x 25mm	18G (1,3mm) x 32mm
REF	09/01/18G/25	09/01/18G/32
COLOR CODE	PINK	PINK
SIZE	18G (1,3mm) x 38mm	19G (1,1mm) x 12,5mm
REF	09/01/18G/38	09/01/19G/12
COLOR CODE	PINK	CREAM
SIZE	19G (1,1mm) x 15mm	19G (1,1mm) x 19mm
REF	09/01/19G/15	09/01/19G/19
COLOR CODE	CREAM	CREAM

SIZE	19G (1,1mm) x 25mm	19G (1,1mm) x 32mm
REF	09/01/19G/25	09/01/19G/32
COLOR CODE	CREAM	CREAM
SIZE	19G (1,1mm) x 38mm	20G (0,9mm) x 12,5mm
REF	09/01/19G/38	09/01/20G/12
COLOR CODE	CREAM	YELLOW
SIZE	20G (0,9mm) x 15mm	20G (0,9mm) x 19mm
REF	09/01/20G/15	09/01/20G/19
COLOR CODE	YELLOW	YELLOW
SIZE	20G (0,9mm) x 25mm	20G (0,9mm) x 32mm
REF	09/01/20G/25	09/01/20G/32
COLOR CODE	YELLOW	YELLOW
SIZE	20G (0,9mm) x 38mm	22G (0,7mm) x 12,5mm
REF	09/01/20G/38	09/01/22G/12
COLOR CODE	YELLOW	BLACK
SIZE	22G (0,7mm) x 15mm	22G (0,7mm) x 19mm
REF	09/01/22G/15	09/01/22G/19
COLOR CODE	BLACK	BLACK
SIZE	22G (0,7mm) x 25mm	22G (0,7mm) x 32mm
REF	09/01/22G/25	09/01/22G/32
COLOR CODE	BLACK	BLACK
SIZE	22G (0,7mm) x 38mm	
REF	09/01/22G/38	
COLOR CODE	BLACK	

Product usage and scope

Safety Huber Needles is intended for infusion or injection of medicinal fluids into patients embedded with subcutaneous infusion port.

This product incorporates an active safety feature that could help to prevent accidental needle stick injuries. The product in 19G, 20G and 22G can be used for high pressure infection process by 325 psi (22 bar), 18G by 200 psi (13 bar). MicroN Safety Needles meet the require- ments of TRBA 250.

Usage method

Product is sterilized by ETO, which is sterile and pyrogen-free, can be used immediately after its package is removed.

- Take the product from its unit package using aseptic technique;
- Connect it to a syringe with appropriate nominal capacity, fill the syringe by drawing medicinal fluid from the vial, then discharge the air;
- Disinfect the human body where the implantable drug delivery system (IDDS) is embedded;
- Remove the protective cap, one hand holds the needle shaft, and the shaft shall be positioned perpendicular to the IDDS. Inspect the patency of infusion passage by repetitive pumping or irrigation, then operate per due operation procedures for injection, infusion.
- Irrigate IDDS per due operation procedures after injection, infusion, then tighten the protective cap after final infusion of the 0.5ml solution. Disinfect the human body where the IDDS is embedded;
- Remove the needle from the puncture site, then activate the safety mechanism by drawing backwards the needle hub in the axial direction opposite to the needle insertion until the needle point is completely covered by the safety mechanism.
- Discard the product in a biohazard container approved for their disposal.

Cautions and Warnings

- Product is prohibited from using in case of damaged primary pack- age, foreign matter, detached protective cap, abnormal needle point status, expired shelf life.
- Take care to avoid the product falling to the ground, Product is for single use only, do not use it repeatedly, please discard it after usage. Handle all medical wastes in accordance with the Medical Waste Regulations or equivalent and avoid needle point injury.
- Product shall be used as per hospital practices, can only be used by professional medical personnel. There shall be no sign of infection in the puncture site for injection before use. Do not re-insert the needle into the infusion port after its removal.

Contradictions

- Do not use for known or potential infection, bacteraemia or septicae- mia related to the medical device.
- Do not use when local tissue cannot be disinfected completely.

Indwelling time

The maximum indwelling time is 7 days. In cases involving cytotoxic agents or irritant substances are used, a reduced dwell time may be required. The appropriate duration of use must be determined by the responsible healthcare professional.

MRI Compatibility information

Safety Huber Needles are classified as MR conditional. No clinical trials have demonstrated the MRI safety of this product. It can be safely used in the MRI environment under the specific conditions listed below.

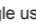
- A static magnetic field of 3 Tesla or less
- A maximum theoretical maximum estimate of whole-body specific absorption rate (SAR) of 2 W/kg (normal operation mode) for 1 hour of scanning
- A spatial gradient field of 49.9 T/m or less
- The maximum commitment coefficient is 149.8 T2/m

Using gradient echo sequences (GRE) in non-clinical tests and in a 3T environment, the maximum artifact width produced by the „ Safety Huber Needles „ is 12.9 mm. In non-clinical testing, Safety Huber Needle produced a temperature rise of 0.9°C at a maximum whole body aver- aged specific absorption rate (SAR) of 2 W/kg for 15 minutes.

Storage and maintenance

Product shall be stored in a clean, cool and dry environment free of any corrosive gas with good ventilation.

Identification, label and description

On the package, „“ means for single use, „“ means product sterilized by ETO.

Validity

Product validity is 5 years. Production lot number and expiry date are demonstrated on the package.

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