Manufacturer / Hersteller /
Fabricant / Fabricante /
Fabricante:

HTL-STREFA S.A.
ul. Adamówek 7, 95-035 Ozorków, Poland
T: +48 42 270 00 10, F: +48 42 270 00 20
www.htl-strefa.com, E: info@htl-strefa.pl

Made in Poland / Hergestellt in Polen /
Fabriqué en Pologne / Hecho en Polonia /
Fabricado na Polónia

0344

EN Safety pen needles
DE Sicherheitspennadeln
FR Aiguilles pour stylo de sécurité
ES Aguja de seguridad para pluma
PT Agulhas de segurança

DropSafe, safety pen needle type 820
820/02/2023

EN

Sterile, single-use safety pen needles intended for use with pen injector devices for the injection of drugs.

For healthcare professionals and lay users.

CAUTION

! Keep out of reach of children without adult supervision.

! Avoid infections: Before making an injection, clean and disinfect the injection site. Always follow your facility sharps injury prevention guidelines or local regulations.

! Do not use if the seal or needle is damaged or missing or if the shield or outer cap needle appears to be damaged.

! Refer to the instructions for use of the drug and pen injector device for specific storage and handling instructions.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the local competent authorities / government agencies, as applicable.

For more information about the device, visit:
<http://htl-strefa.com/products/safety-pen-needles/>

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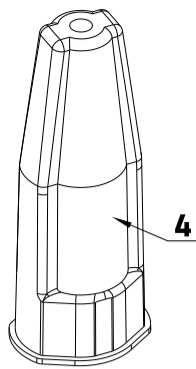
- 1:** Seal
- 2:** Viewing window
- 3:** Shield
- 4:** Outer cap

- Automatic needle lockout after use.
- Visual indication of needle lock out via red stripe.
- Needle safety contained behind shield preventing accidental exposure.
- The needle remains hidden from view increasing patient comfort.
- Viewing window enables checking drug flow through the needle.
- Special wing-shaped design for a comfortable and secure grip.
- Available in three versions: 31G x 5 mm, 31G x 6 mm and 31G x 8 mm.

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Ask your healthcare professional for assistance in choosing the needle length, injection site and technique appropriate for you.
Remember to rotate the injection site!

Read the needle gage (G) and length (mm) on the seal to make sure you are using the right safety pen needle for you or ask for assistance if you have impaired vision.

Also do so, if you are a healthcare professional, a caretaker or family member assisting a patient.
Recommended injection technique: make an injection into a skin vertically at 90° (degree) angle. Alternative injection technique: you may also make a skin fold and/or 45° (degree) angle, if recommended by your healthcare professional.

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
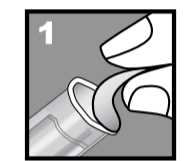
! Check the seal. If it is missing or damaged, use a new safety pen needle.
Remove the seal.
! Check the needle, use a new safety pen needle if the needle appears to be damaged.

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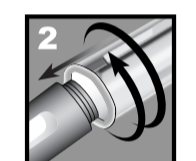
PUSH the safety pen needle straight onto the pen holding it in one axis with the pen and twist until it is tight.
! DO NOT over-tighten.

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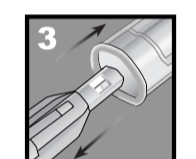Remove the outer cap.
! Make sure not to press the shield before use as any pressure on the shield may cause the needle to lock and the pen needle to be unusable.
! Use a new safety pen needle if the red stripe is visible in the viewing window or the needle appears to be damaged.
! Keep the outer cap.

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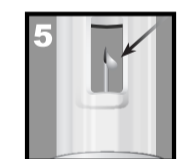
Check if the needle is visible in the viewing window.
! Use a new safety pen needle if the needle is not visible in the viewing window.

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
Perform a priming test if recommended by the pen injector device manufacturer. A drop of liquid should appear on the needle tip visible through the viewing window.
! Use a new safety pen needle if the priming test was unsuccessful.

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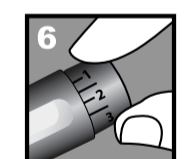
Establezca la dosis prescrita.

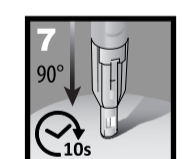Ajuste a dose prescrita.


Insert the needle into a skin according to recommended by your healthcare professional technique in one continuous movement until the shield stops. Deliver the dose.
! DO NOT remove the needle until the dose has been completely delivered. Count to 10 before remove the needle from the skin to prevent from drug leakage. Do not change the direction of the safety pen needle while it remains in the body as this can result in bending or breaking of the needle.

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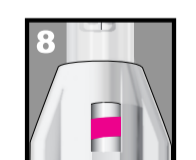
When dose delivery is complete, remove the needle from the skin. The shield will automatically cover the needle until it locks.
! A red stripe will appear in the viewing window. The needle is locked and cannot be reused.
! Note: Liquid may fall back into the safety pen needle which may result in fluid being left on the skin after injection.

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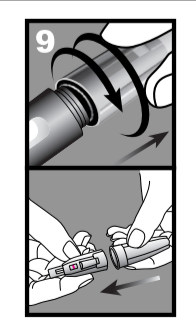Twist off the needle to remove it from the pen injector device.

It is recommended that you cover the back end with the outer cap.

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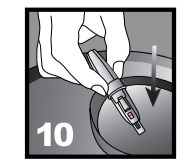
Discard the used safety pen needle into a sharps container.

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