Vliwasorb®

Superabsorbierender Wundverband
Pansement super-absorbant
Superabsorbent Wound Dressing
Apósito de alta absorción

Penso super-absorbente
Medicazione superassorbente
Superabsorberend wondverband
Højabsorberende sårforbinding
Högabsorberande sårförband
Høyabsorberende sårbandasje

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en Instructions for Use

Product Description:
Vliwasorb® superabsorbent wound dressing is a flexible soft wound dressing which is used for the treatment of heavily exuding wounds. In Vliwasorb® superabsorbent dressing the superabsorber (sodium polyacrylate) is imbedded in cellulose and can firmly bind high quantities of wound exudate. As a result of its high absorption capacity, Vliwasorb® superabsorbent wound dressing reduces the number of dressing changes which are necessary when the dressing is soaked, so that the wound can remain undisturbed for longer periods. The wound dressing binds excess exudate and thereby protects the wound environment against maceration. Bacteria that may impair the wound healing process are bound with the exudate. The elastic, perforated wound contact layer made of polyethylene prevents adhesion to the wound. On the side of the dressing which faces away from the wound, Vliwasorb® superabsorbent wound dressing offers a protective blue coloured nonwoven layer which protects patient's clothing and underwear against wound exudate seeping through. Vliwasorb® superabsorbent wound dressing is sterile and individually packed in a pouch. The product is sterilized with ethylene oxide. It is for single use only and may not be resterilized.

Adverse reactions:
None known to date.

Application:
1. Preparation of the wound:
Before applying a new dressing, cleanse the wound as usual (e.g. with sterile saline solution, Ringer's solution) and pat dry the wound edges.
2. Application of the wound dressing:
The size of dressing depends on the wound size; the dressing should overlap the wound by approx. 2-3 cm. Apply the white, elastic side of Vliwasorb® superabsorbent wound dressing onto the wound. Use a suitable material for dressing retention (dressing retention sheet, e.g. Curafix® H, elastic conforming bandage, e.g. Mollelast®).
3. Dressing change:
The intervals in which Vliwasorb® superabsorbent wound dressing must be changed have to be determined by the physician in charge, based on the wound condition (degree of exudation, etc.).

Product composition:
Polyethylene, polypropylene, cellulose, polyethylene terephthalate, ethylene vinyl acetate copolymer, green tea extract, polyacrylate

Indications:
Vliwasorb® superabsorbent wound dressing is indicated for the treatment of heavily exuding wounds such as pressure sores, arterial ulcers, venous leg ulcers, diabetic ulcers, postoperative wounds healing by second intention, laparotomy wounds, on fistulae, and superficial 2nd degree burns.

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Contraindications:
Known hypersensitivity to the product itself or to its components. Do not use in tunnelling wound pockets as the product can expand considerably when wound exudate is absorbed. Do not use on wounds with light exudation, as the wound may dry out and suction pain may occur.

Adverse reactions:
None known to date.

Application:
1. Preparation of the wound:
Before applying a new dressing, cleanse the wound as usual (e.g. with sterile saline solution, Ringer's solution) and pat dry the wound edges.
2. Application of the wound dressing:
The size of dressing depends on the wound size; the dressing should overlap the wound by approx. 2-3 cm. Apply the white, elastic side of Vliwasorb® superabsorbent wound dressing onto the wound. Use a suitable material for dressing retention (dressing retention sheet, e.g. Curafix® H, elastic conforming bandage, e.g. Mollelast®).
3. Dressing change:
The intervals in which Vliwasorb® superabsorbent wound dressing must be changed have to be determined by the physician in charge, based on the wound condition (degree of exudation, etc.).

Please note:
Vliwasorb® superabsorbent wound dressing may not be cut or torn. The product is sterile unless the package is opened or damaged.

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