Vliwasorb® Pro

Superabsorbierender Wundverband
Pansement super-absorbant
Superabsorbent Wound Dressing
Apósito de alta absorción

Compressa superabsorvente para feridas
Medicazione superassorbente
Superabsorberend wondverband
Højabsorberende sårforbinding
Högabsorberande sårförband
Høyabsorberende sårbandasje

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en Instructions for Use

Product description:
Vliwasorb® Pro is a flexible and soft superabsorbent wound dressing used for acute and chronic wounds with very high to moderate amounts of exudate. Once the exudate is absorbed, the superabsorbent core, which has a very high absorption capacity, forms a gel and retains the excess exudate in the wound dressing, thereby protecting the wound and the wound environment from maceration. Vliwasorb® Pro supports physiological wound healing. An inner nonwoven layer helps distribute the exudate evenly and transfer it rapidly to the superabsorbent core. Due to its very high absorption capacity, Vliwasorb® Pro reduces the need for frequent dressing changes as a result of soaking through, thereby allowing the wound to remain undisturbed for longer. Its ergonomic shape and wide edge make it easy to apply, especially on parts of the body that are difficult to treat, and they provide high wearing comfort even under compression. The soft, elastic polyethylene wound contact layer reduces the risk of adhesion to the wound. The outer layer of the dressing is covered with a blue nonwoven material that protects the patient's garments and bed linen. Odour formation is reduced while the wound is being treated. Vliwasorb® Pro is individually sealed and sterile (sterilised by ethylene oxide). The product is intended for single use and must not be resterilised. The product is sterile as long as the packaging is unopened and undamaged.

Contraindications:
Known hypersensitivity to the product or its components. Not for use in tunnel-forming wound pockets, as the product may expand considerably with the absorption of wound exudate. Not for use on dry wounds.

Side effects:
In very rare cases skin irritation or allergies may develop.

Use:
1. Preparing the wound:
Cleanse the wound as usual prior to applying the new dressing (e.g. with sterile saline solution, Ringer's solution) and pat the wound edges dry.
2. Applying the wound dressing:
The size of the dressing should be based on the wound area; the dressing should overlap the wound by approx. 3 cm. Apply the white side of Vliwasorb® Pro to the wound. In case of deep wounds, an appropriate wound filler may be used, depending on wound status. The dressing must be fixed with a suitable material (retention dressing, e.g. Curafix® H, elastic conforming bandage, e.g. Mollelast® or in treatment of underlying diseases with compression bandages, e.g. Rosidal®).
3. Changing the wound dressing:
The treating physician will determine the dressing change intervals for Vliwasorb® Pro, depending on the wound status and degree of exudate from the wound.

Product composition:
Polyethylene, polypropylene, cellulose, polyethylene terephthalate, ethylene vinyl acetate copolymer, polyacrylate

Indications:
Vliwasorb® Pro is suitable for the care of acute and chronic wounds with very high to moderate amounts of exudate, such as:
• pressure ulcers, arterial ulcers, venous lower leg ulcers, diabetic ulcers
• postoperative wounds healing by secondary intention
• post-traumatic wounds

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