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Dear Customer,

we are pleased that you have chosen a product from our range.

Please read these operating instructions carefully and carefully and keep them accessible for other users. Please pay attention to the application instructions and notes.

With kind regards

Your Novidion Team

Summary

This blood pressure monitor is an aneroid sphygmomanometer that measures using the Riva-Rocci and Korotkoff method. In this process, an artery above the elbow bend is clamped by building up pressure. When the pressure is released, the upper blood pressure value (systolic blood pressure) is calculated. After a complete blood flow through the artery, further pressure is released and the lower blood pressure (diastolic blood pressure) is calculated.

Read the instructions carefully and completely before use and follow the instructions exactly.

- Familiarize yourself with the device before using it.
- Do not measure your blood pressure immediately after physical exertion or when you are excited.
- If you are measuring yourself, relax for about 15 minutes before taking the measurement to avoid inaccurate measurements.
- To ensure correct measurement results, use a suitable cuff size.

Instructions for measuring blood pressure

1. Preparation (instructions for right-handed people):

- Applying the cuff:

Take off your outer clothing or roll up your sleeves so that your arm is as free as possible. Open the Velcro fastener and pull the deflated cuff on your left arm up to above your elbow (the lower edge of the cuff should be 2-3 cm above the crook of your arm). The tubes must point downwards. Ideally, you should use the cuff's artery indicator as a guide. This must be centered on the crook of the upper arm, close to the brachial artery (arteria brachialis). You can find this by relaxing your arm and gently pressing with your fingertips until you feel a pulse.

Tighten the Velcro fastener, but not too tightly; there should be about a finger's width of space between the cuff and the arm.

- Attaching the stethoscope:

Separate stethoscopes are required for blood pressure monitors of this type. Connect the stethoscope tubes to the stethoscope and insert the ear tips into the ears. The chest piece of the stethoscope must be placed on the artery below the cuff.

- Positioning:

The arm with the cuff must be approximately at the same level as the heart. If you are taking the measurement while sitting, place your forearm slightly extended on a table in front of you. If you are taking the measurement while lying down, place your arm directly next to you.

2. Build up pressure

- Close the bleed valve on the hand pump by turning the screw clockwise. Do not overtighten the screw, so that you can open it again later!
- To pump up the cuff, squeeze the rubber bulb repeatedly and quickly.
- Inflate the cuff until the gauge reads approximately 20-30 mmHg ABOVE the normal systolic value. If the systolic value is unknown, inflate until the blood flow sound is no longer audible, but do not inflate above 300 mmHg.

3. Releasing pressure

- Open the bleed valve slowly by turning the screw anticlockwise. The pressure drop should be approximately 3 mmHg per second (this corresponds to a drop of 1-2 lines per second on the scale).
- ⚠ Please note: The speed of pressure release affects the accuracy of the measurement result. The pressure should not be released too quickly, as you may miss the correct time to take the reading. If the pressure is released too slowly, the arm is without blood supply for too long because the pressure interrupts blood flow.

4. Determine systolic and diastolic pressure

- Listen for the pulse with the stethoscope throughout the pressure release. As soon as you hear the first tone, read the value on the scale and note it down. This is the upper blood pressure value (systolic pressure).
- As you release the pressure further, the pulse sounds become quieter until they disappear completely. Note the last value at which you could hear the pulse. This is the lower (diastolic) blood pressure.

5. Completing the measurement

- Fully open the air release valve to allow all the pressure to escape from the cuff.
- Remove the cuff from your arm and place the blood pressure monitor back in its packaging or the carrying bag.

Interpretation of the values and possible sources of error

The determined values are rounded up to the nearest five or ten. The overview provided by the World Health Organization (WHO) provides a guide to interpreting the values. However, if in doubt, you should not rely on these assessments, but always consult a

doctor if you suspect a medical problem.

Description	SYS value (mmHg)	DIA value (mmHg)
Optimal	< 120	< 80
Normal	120 - 129	80 - 84
High Normal	130 - 139	85 - 89
Hypertension Grade 1	140 - 159	90 - 99
Hypertension Grade 2	160 - 179	100 - 109
Hypertension Grade 3	>/= 180	>/= 110

Please also note that manual measurements are always prone to application errors. Therefore, please be aware of the following sources of error and try to avoid them:

Values too low:	Values too high:
Clothing not removed from upper arm	Cuff too small
Cuff too large	Cuff not wrapped tightly enough
Cuff not pumped up enough (SYS)	Air released too quickly (DIA)
Air released too quickly (SYS)	Repeated pumping on the same arm
Arm positioned above heart level	Interfering noises (tones not clearly audible)
Stethoscope pressed on too hard	
Interfering noises (tones not clearly audible)	

Further information, maintenance, cleaning:

When used and stored correctly, this device will serve you well for many years. Please observe the following maintenance and cleaning instructions:

- Always clean the blood pressure monitor with a damp, clean, soft cloth.
- Do not use chemicals.
- Do NOT immerse the device in liquids.
- Do not sterilize the device.
- Handle the blood pressure monitor with care, do not expose it to shocks, impacts or other physical influences. Do not drop the device!
- Do not expose the device to excessively high or low temperatures.
- Always release all pressure from the cuff before storing the device.
- Protect the surface of the stethoscope membrane from damage.
- Do not pump more than 300 mmHg.
- Make sure that the hose system has no leaks.
- The instrument must be calibrated by a certified professional every 2 years.

Technical data

Model name: KTJ-20

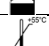
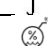
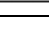
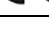
Measuring graduation: 2mmHg

Measuring range: 0-300 mmHg

Measuring accuracy: ± 3 mmHg

The lowest value on the scale is "0", the aneroid pointer on the "0" position must be within a tolerance of ± 3 mmHg.

The Aneroid Sphygmomanometer complies with the EN1060 guideline for blood pressure monitors

Symbol	Meaning	Symbol	Meaning
	Caution! Please refer to the accompanying document (user manual)		Caution! Please refer to the accompanying document (user manual)
	Recyclable		Batch number
	Expiry date		Store in a dry place
	Storage temperature limit		Restriction on storage humidity
	Manufacturer		European representative
	Date of manufacture		Serial number
	Medical device		This product complies with the Medical Device Directives (EU) 93/42/EEC and 2007/47/EEC.

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Novidion GmbH
Lotus NL B.V.
Wuxi Exanovo Medical Instrument Co., Ltd.