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Produktbeschreibung

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en Instructions for Use

Product description

Viwazell® Pro is a soft absorbent dressing consisting of a nonwoven contact layer, a nonwoven distribution layer, an absorbent core and a clothing protection layer. The nonwoven distribution layer helps to distribute the exudate from the wound area and transport it to the absorbent core.* The absorbent body consists of cellulose flakes and a superabsorbent polymer core, so Viwazell® Pro has a very high absorption capacity. By binding excess exudate in the absorbent body, Viwazell® Pro reduces the risk of maceration of the wound and wound area.* The very high absorption capacity allows long dressing change intervals and long periods of wound rest.* Furthermore, on the side away from the wound, Viwazell® Pro features blue nonwoven material that protects the patient's clothing and bed linen. Viwazell® Pro is individually sterile packed, sterilised with ethylene oxide.

Product composition

Viscose, polyamide, cellulose, polyéthylène, polyéthylène téréphthalate, ethylene vinyl acetate copolymer, polyacrylate, polypropylene, hotmelt glue

Intended use

Absorbent multilayer wound dressing indicated for exudate management of acute and chronic wounds.

Indications

Viwazell® Pro is suitable for the care of superficial acute and chronic wounds

Indications

moderate to very high amounts of exudate

Indications

and that are superficial, in

- arterial and venous ulcers
- diabetic ulcers
- pressure ulcers
- post-traumatic wounds

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Warnings and precautions

Do not cut or tear Viwazell® Pro. The product is for single use by one patient only and may not be resterilised. The product is sterile as long as the packaging is unopened and undamaged. Do not reuse Viwazell® Pro if the package is damaged or accidentally opened.

Instructions for use

Application is performed by healthcare professionals.

1. Preparation

Before wound assessment and application of a new dressing, any dressing residue should be removed from the wound bed when cleansing the wound (e.g. with sterile saline solution, Ringer's solution). The surrounding skin should be clean and dry.

2. Application

The size of the dressing depends on the wound size. The dressing must overlap the wound by approx. 1-2 cm. Apply the white side of Viwazell® Pro to the wound to be treated. In the case of deep wounds, Viwazell® Pro can be used with a wound filler appropriate for the particular wound status. Viwazell® Pro must be retained with a suitable material (Curafix® H Retention dressing, Molelast® Elastic conforming bandage or, as part of treatment of underlying diseases, with Rosidal® Compression bandages).

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