EN

Plaster/Adhesive bandage

Intended Purpose: For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instructions for use: Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage/plaster. Not suitable for children under 3 years old. In rare cases, a reaction to the adhesive bandage/plaster may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases consult a doctor immediately. If you are diabetic consult your doctor before use.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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© Copyright. All rights reserved by RFX+Care International A/S NF Textile plaster-IFU 2025-11
RFX+CARE Manufacturing Co., Ltd. No. 599 Wuxie Road, Yuecheng District, Shaoxing, Zhejiang, China 312000
RFX+CARE International A/S Gammel Strandvej 22B, 2990 Nivå, Denmark
CMC Medical Devices GmbH Rigistrasse 3, 6300 Zug, Switzerland