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Evolsin® Anti-Ohrenschmalz

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EN Instructions for use

Evolsin® Anti-Earwax

Medical device for the mechanical removal of excess earwax

Product description and intended purpose

Evolsin® Anti-Earwax is a mechanical medical device designed for the gentle and safe removal of excessive earwax (cerumen) from the external auditory canal, helping to prevent or relieve earwax blockage and the associated hearing impairment, pressure sensations, and ear inflammation. Evolsin® Anti-Earwax may also be used for regular ear hygiene and care.

Indications

Excessive earwax (cerumen), earwax blockage, excessive earwax production

Product components

Evolsin® Anti-Earwax consists of:  
- an ergonomic handle made of high-quality plastic,  
- a flexible, interchangeable cleaning head with a spiral tip or brush.

Principle of operation

The spiral tip removes earwax through a mechanical rotating motion that gently loosens and guides the cerumen out of the external auditory canal without pushing it further towards the eardrum. The application is intended to support the ear's natural self-cleaning function.

Application

Preparation:  
- Wash your hands thoroughly with soap and water.  
- Check the product for visible dirt or damage. Use only clean, intact tips.

Use the different attachments

The Evolsin Anti-Earwax set contains various interchangeable attachments suitable for different applications. All attachments are intended for the mechanical removal of excessive earwax from the external auditory canal.

Spiral attachment (Fig 1)

Recommended for:  
- The targeted removal of firmer or deeper-sitting earwax  
- Intensive cleaning in case of heavier contamination  
- Users with previous experience in ear cleaning

Application:

Slowly insert the spiral attachment into the external auditory canal and gently rotate clockwise. The spiral shape transports excess earwax outward. Do not apply pressure.

b. Silicone brush attachment (Fig 2)

Recommended for:  
- Regular cleaning of the external auditory canal  
- People with sensitive skin or a narrow ear canal  
- Loosening and removing soft or fresh earwax

Application:

Carefully insert the brush attachment into the external auditory canal and gently rotate it. The soft silicone nubs help to loosen and gently remove loose earwax without irritating the skin.

Instructions for use of all attachments:

- Only use in the external auditory canal – never insert deeply!  
- Do not use if you have an injury, pain or inflammation in your ear.  
- Clean attachments thoroughly with warm water after each use and allow to dry completely.  
- Regularly check attachments for damage and replace if necessary.

Application steps:

1. Select a clean, undamaged cleaning tip and securely it to the handle.  
2. Carefully insert the tip into the external auditory canal – no deeper than 1 cm.  
3. Do not drill or press with force.

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5. Clean the outer ear canal with a soft, clean cloth or water.  
6. After use: clean the tip under warm water or dispose of it according to section "Disposal".

Recommended use

1–2 times per week, or as individually required.  
More frequent use is unnecessary and may irritate the skin of the ear canal.

Contraindications

Do not use in case of:  
- Injuries or inflammation of the ear or ear canal  
- Perforated eardrum  
- Ear pain, discharge, hearing loss, or dizziness  
- After ear surgery - Children under 3 years of age  
- Allergy to silicone.

If symptoms or pain are unclear, consult a physician before use.

Patient and user group

For self-use by persons aged 3 years and older. For children, use only on medical advice and under adult supervision.

Warnings and precautions

**Safety-related information:**  
- Do not insert deeper than 1 cm into the auditory canal.  
- Do not apply excessive pressure.  
- Do not use if the tip is damaged, hard or dirty.  
- Do not share attachments with other persons to avoid cross-contamination.  
- If pain, dizziness, bleeding, or persistent pressure occurs, discontinue use immediately and consult a physician.

Not intended for use on animals. Only use as intended.

Side effects

Improper use may cause:  
- Mild irritation or redness of the ear canal  
- Minor bleeding if inserted too deeply  
- Discomfort or pressure sensation

If symptoms or pain are unclear, consult a physician before use.

Notice for patients/users

Serious incidents related to the product must be reported to the manufacturer and the competent authority of the EU Member State in which you live.

Cleaning and maintenance

- Rinse the cleaning tip thoroughly under warm running water after each use.  
- Dry with a clean cloth and allow to air dry.  
- Do not treat with aggressive cleaners or alcohol.  
- Regularly inspect reusable tips for wear and replace if necessary. - Store hygienically

Storage

- Store in a dry and clean place.  
- Protect from direct sunlight and heat.  
- Keep out of reach of children.

Disposal

- Dispose of used or damaged cleaning tips with household waste.  
- Do not dispose of the product in the toilet or wastewater.  
- Dispose of packaging materials according to local recycling regulations.

Classification

Class I medical device (non-sterile) according to EU Regulation (EU) 2017/745 (MDR).

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