IQCATH

IQ3104.xx

IQ3114.xx

Gebrauchsanweisung	Deutsch	2
Instructions for use	English	4
Notice d'utilisation	Français	6
Istruzioni per l'uso	Italiano	8
Gebruiksaanwijzing	Nederlands	10
Bruksanvisning	Svenska	12
Brugsanvisning	Dansk	14
Käyttöohje	Suomi	16
Bruksanvisning	Norsk	18
Notkunarleiðbeiningar	Ísland	20
Kasutusjuhised	Eesti	22
Instrucciones de uso	Español	24
Instruções de utilização	Português	26
Οδηγίες χρήσης	Ελληνική	28
Használati utasítás	Magyar	30
Instrukcja użytkowania	Polski	32
Návod k použití	Čeština	34
Instrucţiuni de utilizare	Română	36
Инструкция за употреба	Български	38
Navodila za uporabo	Slovenščina	40
Upute za uporabu	Hrvatski	42
Uputstvo za upotrebu	Српски/srpski	44
Uputstvo za upotrebu	Bosanski	46
Упатство за употреба	Македонски	48
دستور العمل استفاده	فارسی	50
قسطرة التبول الخارجي ذاتية الالتصاق	العربية	52

Deutsch

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Single use gel catheter for ISC (Intermittent self-catheterisation)

- Ready for use
- The catheter with 2 functional zones (SAUER-rounded tip and guiding zone)
- Functional packaging for variable use
- Total length: ca. 430 mm, Length without funnel: ca. 400 mm
- 4 sizes: CH 10, CH 12, CH 14, CH 16

Group of Users: IQ3104.xx, IQ3114.xx – Men

Intended use:

The IQ-Cath is a single use product designed for temporary drainage of urine from the urinary bladder in a person with bladder voiding dysfunction for therapeutic or diagnostic purposes. It is suitable for intermittent self-catheterisation (ISC) or carer-assisted clean intermittent catheterisation (CIC).

Indication:

The indication for ISC or CIC is made following relevant diagnostic tests and is dependent on the individual's situation.

The IQ-Cath is suitable for catheterisation in the following situations: bladder voiding dysfunction of various origins (e.g. neurogenic), underactive detrusor or acontractile detrusor, chronic urinary retention, abnormal micturition (straining) with or without residual urine.

Contraindication:

The IQ-Cath may be unsuitable in the following cases, and a health care professional must be consulted before use: injury of the urethra, bleeding of the urethra, acute inflammation of the urethra or prostate, obstructions (e.g. tumours) in the urethra, at the sphincter or at the prostate blocking passage.

Instructions for use:


The first use should be supervised by a professional.
If liquid has already leaked from the gel sachet into the packaging, the catheter may no longer be used.
With gel catheters, there may sometimes be a slight delay after insertion before the urine begins to flow.

Risks and Cautions:


The use of catheters (especially improper use) may cause irritation/infection of the urinary tract. You must seek medical attention if this occurs.
Improper use may cause residual urine to remain in the bladder. This catheter is intended for single use only. The risk of infection increases if the catheter is reused or reprocessed. Do not use without suitable sterile lubricating gel as it is not lubricated. Do not use if the sterile packaging or the catheter is damaged as there is a risk of infection. We recommend an operating temperature range of 15-25°C.
Any serious adverse event occurring during use of IQ-Cath must be reported to the manufacturer and the competent authority of the user's country of residence.

	Catalog number		 0123 CE marked medical device
	Serial number		Use by (year-month-day)
	Store in a dry place		Not for reuse
	This way up		Pharmaceutical central no.
	Do not resterilise		Drug Tariff
	Quantity		List of products and services
	Keep away from sunlight		Do not use if package is damaged
	Sterilised with ethylene oxide		Date of manufacture
	Single sterile barrier system		Manufacturer
	Single sterile barrier system with protective packaging outside		Swiss authorised representative
	Batch number		UK authorised representative
	Observe instructions for use		SAUER-rounded tip
			Ready for use


Fig. 1-7: Avoid touching any part of the catheter surface (except for the connector) directly. You can use the functional packaging as a gripping aid.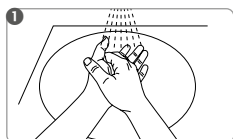


Before use, wash hands thoroughly and disinfect them.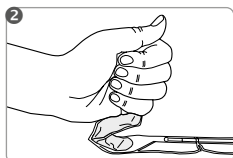


Open the gel sachet by folding and squeezing. Make sure the whole content of the sachet is emptied into the packaging.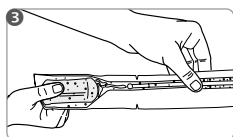

Spread the sterile lubricating gel along the full length of the catheter (except for the funnel) to make sure all of the surface is coated with gel.

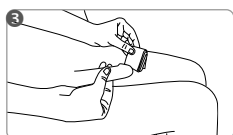
Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).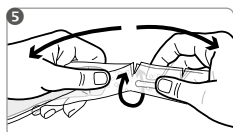


Use tear marks at the front end to open the functional packaging and then tear off with a jerk including the gel sachet.


Using the functional packaging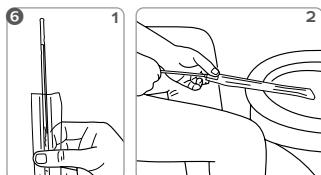

A) The functional packaging can be used as an extension for draining the urine directly into the toilet. To do so, tear off the functional packaging along the front tear marks (catheter tip). Then slide the catheter forwards (Fig. 6.1) until the connector seals the packaging (Fig. 6.2).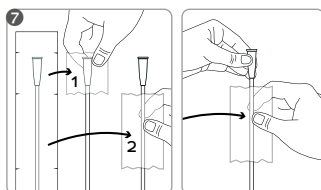

(B) The functional packaging can also be used as a short or long gripping aid (1 or 2). To do so, remove the required section of film along the corresponding tear marks.

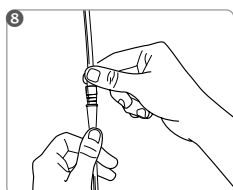Alternatively, you can hold the catheter at the connector and remove the functional packaging completely. Attach a urine bag to the catheter if necessary. If the catheter becomes stuck inside the packaging, twist the catheter slowly to remove. Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly. Dispose of the catheter in the general household waste.

MANFRED SAUER GmbH does not accept responsibility for damages or injuries that may result from improper handling contrary to these instructions for use or from neglecting the hygiene guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser: MANFRED SAUER GmbH · Neurott 7-15 · 74931 Lobbach · Germany · +49 6226 960100

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