Verbandschuhe / Spezialschuhe für die Kurzzeitanwendung
Bandage shoes / special shoes for short-term application
Chaussures thérapeutiques / spéciales à usage temporaire
Scarpe ortopediche / scarpe speciali per l'utilizzo di breve durata
Zapatos ajustables / zapatos especiales para aplicaciones breves
Verbandschoenen / speciale schoenen voor kortstondig gebruik
Sapatos terapêuticos / Sapatos especiais para o uso temporário
Bandašjesko / spesialsko for kortvarig bruk

Modelle / Models / Modèles / Modelli /
Modelos / Modellen / Modelos / Modell :

**Pantomobil D, Theramed D1, Theramed D3,
Sanicabrio, Sanicabrio DXL, Sanicabrio LXL,
Sanisoft D, Sanisoft,
München 2 LXL, München 3 (braun/schwarz),
Wallgau, Grainau, Flex 2,
Theralight 1, GentleWalk-Hi, GentleWalk-Lo**

DE

EN

FR

IT

ES

NL

PT

No

ALLGEMEINES

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NO

GENERAL INFORMATION

Before using the bandage shoe, please read this instruction manual carefully. Intended application and use can only be ensured if all instructions, information and procedures described in this manual have been thoroughly read and understood. In case of side effects (e.g. skin swelling, bruises, disturbance of blood circulation, etc.), please contact your physician. In case of any uncertainty regarding something in the instruction manual or unclear information, general or safety instructions, please contact your specialist dealer or us directly before using the bandage shoe. This applies in particular for safety instructions.

Incorrect use and non-compliance with general or safety instructions found in this instruction manual may fully or partly compromise the function, the treatment efficiency and the protective effect of the bandage shoe and lead to a risk of injury (including to third persons) or property damage. This aid is to support healing and must always be used in compliance with the instructions of your physician or a medical professional (or after consultation). The manufacturer does not assume any responsibility for this. Wearing of socks or bandages inside the shoe is recommended. This aid is only intended for use by a single patient.

In case of severe incidents (major injury) during use of the product, the following persons and offices must be informed:

1. The manufacturer
2. In Germany and outside of the European Economic Area:
The responsible German Federal Institute for Drugs and Medical Devices.
3. In other countries in the European Economic Area, the respective responsible authorities.

SAFETY INSTRUCTIONS

Please observe the safety information as optimum effect of the shoe can only be achieved by correct application.

The bandage shoe may only be used if it is in perfect, undamaged condition, if the padding is not damaged and with the fasteners.

This must be checked by the user before every use.

Keeping open individual or several fasteners and extensive loosening of the straps during use of the shoe reduces the therapeutic effect and may cause injuries.

The shoe must not be worn on open wounds.

The shoe should fit firmly and not too tight to ensure that the blood circulation is not restricted.

Avoid a loose or excessively tight fit of the shoe.

The shoe is only intended for use by one person.

SCOPE OF DELIVERY

- 1 pair of bandage shoes
- Instruction manual
- Packaging

DE

EN

FR

IT

ES

NL

PT

NO

INTENDED USE AND INDICATIONS

Thanks to the hook and loop fastener, the shoe has an extra-wide opening for putting it on and taking it off. Intended for use with swollen feet, hammer toes, after surgeries and with plasters and bandages. Can be used post-operatively and in case of other (not open) wounds and in combination with extensive and padded bandages during long healing processes to enable at least partial loading of the foot for walking including outdoors. Additional padding is not required. Additionally, the shoe features a neuropathic zone in the forefoot section of the sole to prevent any hazardous "BRAKE EFFECTS" when dragging the feet. The shoes are suitable for indoor and outdoor use. They must not be used outdoors in case of ice and packed snow and heavy rain.

CONTRAINDICATIONS

Currently, not known.

ADJUSTMENT

All our bandage shoes / special shoes can be individually adjusted as follows by qualified orthopaedic personnel in orthopaedic service centres:

- Selection of suitable models
- Selection of the right size (36 to 47)
- Smoothing of the outsole at the front
- Removable insole for adjustment and change
- Cutting to length of textile fasteners and upper material and reattachment
- Readjustment by hook and loop fasteners
- Inserting of rigid plates into the sole

USE

General

Instep and heel area.

Open the forefoot and/or heel fasteners and place the foot in a comfortable position. Afterwards, close all fasteners in such a way that the shoe is tight but not too tight.

Pantomobil D		Theramed D1	
MULTIFUNCTIONAL, WIDE-OPENING SPECIAL SHOE IN THE FORM OF SLIPPERS made of soft textile material with a wide hook and loop fastener at the forefoot and a removable fastener at the heel. - Open toe area - Open heel		MULTIFUNCTIONAL, WIDE-OPENING SPECIAL SHOE IN THE FORM OF SANDALS made of soft textile material with double hook and loop fastener at the forefoot. - Open toe area - Closed heel Thanks to its double hook and loop fastener, Theramed D1 enables optimum adjustment in width at the forefoot and instep	
Thanks to its removable hook and loop fastener at the heel, Pantomobil D enables individual adjustment to personal preferences. At the forefoot, the shoe can be adjusted in width with a wide hook and loop fastener.			

DE

EN

FR

IT

ES

NL

PT

NO

USE

<p>Theramed D3</p> <p>MULTIFUNCTIONAL, WIDE-OPENING SPECIAL SHOE IN THE FORM OF SANDALS made of soft textile material with double hook and loop fastener at the forefoot. - Open toe area - Open heel</p> <p>Thanks to its triple hook and loop fastener, Theramed D3 enables optimum adjustment in width at the forefoot and instep.</p>		<p>Sanicabrio</p> <p>ANKLE-HIGH SPECIAL SHOE WITH SINGLE HOOK AND LOOP FASTENER made of soft textile material with closed heel. Thanks to its hook and loop fastener, Sanicabrio enables wide opening of the shoe at the forefoot.</p>	
<p>Sanicabrio DXL</p> <p>ABOVE-ANKLE-HIGH, EXTRA-WIDE SPECIAL SHOE made of soft textile material with wide-opening "Cabrio" fastening system at the forefoot and with hook and loop fastener at the heel to slip on the shoe super easily.</p>		<p>Sanicabrio LXL</p> <p>LOW, EXTRA-WIDE SPECIAL SHOE made of soft textile material with wide-opening "Cabrio" fastening system at the forefoot and with hook and loop fastener at the heel to slip on the shoe super easily.</p>	
<p>Sanisoft D</p> <p>LOWER, WIDE-OPENING SPECIAL SHOE made of soft textile material with double hook and loop fastener at the forefoot and a single hook and loop fastener at the heel. Thanks to its double hook and loop fastener, Sanisoft D enables optimum adjustment in width at the forefoot and instep.</p>		<p>Sanisoft</p> <p>LOW, EXTRA-WIDE SPECIAL SHOE made of soft textile material with double hook and loop fastener at the forefoot. Thanks to its double hook and loop fastener, Sanisoft enables optimum adjustment in width at the forefoot.</p>	
<p>München 2 LXL</p> <p>EXTRA-WIDE SPECIAL SHOE MADE OF A STRETCH AND MICRO VELOUR COMBINATION IN THE FORM OF LOW SHOES with 2 hook and loop fasteners at the forefoot. Increased forefoot volume made of 4-fold elastic material.</p>		<p>München 3</p> <p>WIDE SPECIAL SHOE MADE OF A STRETCH AND DACRON COMBINATION IN THE FORM OF LOW SHOES with hook and loop fastener at the forefoot.</p>	
<p>Wallgau</p> <p>WIDE SPECIAL SHOE MADE OF A ELASTANE AND STRETCH COMBINATION IN THE FORM OF LOW SHOES with hook and loop fastener at the forefoot and 2 insoles. Increased forefoot volume made of elastic material.</p>		<p>Grainau</p> <p>WIDE SPECIAL SHOE MADE OF A ELASTANE AND STRETCH COMBINATION IN THE FORM OF LOW SHOES with hook and loop fastener at the forefoot and 2 insoles. Increased forefoot volume made of elastic material.</p>	

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USE

Flex 2		Therelight 1	
<p>MULTIFUNCTIONAL WIDE SPECIAL SHOE MADE OF A NEOPRENE AND ELASTANE STRETCH COMBINATION IN THE FORM OF SLIP-ON SHOES with hook and loop fastener at the forefoot and pivoting/removable strap at the heel. Increased forefoot volume made of elastic material. The heel strap features a hook and loop fastener for width adjustment and can be pivoted to the front or removed at the push buttons as necessary.</p>		<p>MULTIFUNCTIONAL, WIDE-OPENING SPECIAL SHOE IN THE FORM OF SANDALS with wide hook and loop fastener at the forefoot. - Open toe area - Closed heel</p> <p>Thanks to its removable hook and loop fastener at the heel, Therelight 1 enables individual adjustment to personal preferences. At the forefoot, the shoe can be adjusted in width with a wide hook and loop fastener.</p>	
GentleWalk-Hi		GentleWalk-Lo	
<p>ANKLE-HIGH, EXTRA-WIDE SPECIAL SHOE with double hook and loop fastener at the forefoot and wool lining to keep the feet warm. Thanks to its double hook and loop fastener, GentleWalk-Hi enables optimum adjustment in width at the forefoot and instep.</p>		<p>LOW, EXTRA-WIDE SPECIAL SHOE with double hook and loop fastener at the forefoot and wool lining to keep the feet warm. Thanks to its double hook and loop fastener, GentleWalk-Lo enables optimum adjustment in width at the forefoot and instep.</p>	

TECHNICAL DATA

Sizes*: 36, 37, 38, 39, 40, 41, 42, 43, 44, 45, 46, 47

*) For precise width, length and height dimensions, please refer to the separate data sheet of our models.

	Last width	Weight (size 43)	Materials			Outer sole
			Upper material	Lining	Insole	
Pantomobil D	Extended width J	277 g	Polyester, Dacron	Polyester	Cotton, EVA	PU
Theramed D1	Extended width J	278 g	Polyester, Dacron	Polyester	Cotton, EVA	
Theramed D3	Extended width J	280 g	Polyester, Dacron	Polyester	Cotton, EVA	
Sanicabrio	Comfortable width G	290 g	Polyester, Dacron	Polyester	Cotton, EVA	
Sanicabrio DXL	Extended width J	292 g	Polyester, Dacron	Polyester	Cotton, EVA	
Sanicabrio LXL	Extended width J	288 g	Polyester, Dacron	Polyester	Cotton, EVA	
Sanisoft D	Extended width J	282 g	Polyester, Dacron	Polyester	Cotton, EVA	
Sanisoft	Comfortable width G	305 g	Polyester, Dacron	Polyester	Cotton, EVA	
München 2 LXL	Extended width J	280 g	Polyester, Dacron, neoprene	Polyester	Cotton, EVA	
München 3 (brown/black)	Comfortable width G	305 g	Polyester, Dacron, neoprene	Polyester	Cotton, EVA	
Wallgau	Comfortable width G	314 g	PU, elastane	Polyester	Cotton, EVA	
Grainau	Comfortable width G	312 g	PU, elastane	Polyester	Cotton, EVA	
Flex 2	Extended width J	295 g	Neoprene, elastane	Polyester	Cotton, EVA	
Therelight 1	Comfortable width G	296 g	Polyester, mesh	Polyester	Cotton, EVA	
GentleWalk-Hi	Comfortable width G	308 g	Polyester, mesh	Wool	Wool, EVA	
GentleWalk-Lo	Comfortable width G	321 g	Polyester, mesh	Wool	Wool, EVA	

DE

EN

FR

IT

ES

NL

PT

NO

CARE INSTRUCTIONS

Can be machine-washed at 30 °C

Do not use bleach

Not suitable for driers

Do not iron

Do not dry-clean

The shoes can be disinfected using any commercially available or common household disinfectant.

MAINTENANCE

The product is maintenance-free

DECLARATION OF CONFORMITY

The requirements of EU Regulation 2017/745/EU for medical devices are fulfilled. The CE mark refers to EU Regulation 2017/745/EU. Device classification: Class I.

DISPOSAL

Bandage shoes can be disposed of as residual household waste. For hygienic and orthopaedic reasons, bandage shoes should not be reused nor recycled after expiry of the service life or at the end of therapeutic treatment.

WARRANTY CONDITIONS

This product was produced with utmost care and thoroughly checked before leaving the factory. For this reason, we provide a guarantee of 24 months as of the date of purchase for this product. **Promed** products comply with their description and respective specifications. It is your responsibility to ensure that the products purchased by you are suitable for the application intended by you.

- In case of demonstrable defects in material and/or workmanship during correct use and identified within the guarantee period, we will replace all defective parts of the product free of charge including labour costs for repair under guarantee.
- **Guarantee does not include:**
 - Normal wear of the product. Defects caused by transport or storage of the product. Defects or damage caused by incorrect use or insufficient care. Damage caused by non-compliance with the information in the instruction manual. Damage caused by unauthorised modification of the product not performed by **Promed**.
 - Damage caused by sharp objects, torsion, compression, falling, unusual impacts or other events beyond the reasonable control of **Promed**.
 - Wear parts (e.g. moving parts such as ball bearings, etc, fasteners) are generally excluded from guarantee.

DE

EN

FR

IT

ES

NL

PT

NO

WARRANTY CONDITIONS

- **Claims under guarantee cannot be made if:**

the product is not returned in its original packaging or another adequate and secure packaging; it was modified or repaired by third parties or companies other than **Promed** or a dealer authorised by **Promed**; the product was not repaired with spare parts approved by **Promed**; the serial number / batch number of the product was removed, erased changed or made illegible.

- For hygienic reasons, any products getting into contact with the body or bodily fluids (e.g. blood) must be packaged in a separate plastic bag before return. In these cases, a special note must be made in the package or in accompanying documents.
- For any components repaired during the guarantee period or replaced products, the guarantee only applies for the remaining original guarantee period, provided that replacement or repair was performed by **Promed** or a dealer authorised by **Promed**.
- The guarantee period starts on the day of purchase. Guarantee claims must be made within the guarantee period. Any complaints made after expiry of the guarantee period cannot be taken into account. Within these guarantee conditions, the guarantee only applies if a receipt or similar document is presented for verification of the purchase date. Technical and optical changes as well as changes to the kit are reserved!

This guarantee is only valid and can be legally enforced in the country in which the product was purchased by the user, provided that offer of the product for sale in this country was within the intention of **Promed**. This guarantee can also be enforced in any country in the European Economic Area in which **Promed** has an authorised importer or sales partner. Depending on the respective country, special and deviating guarantee and warranty conditions may apply due to applicable laws. These laws are neither excluded nor limited by these guarantee conditions. If admissible under national law, the guarantee period will not be extended, renewed or otherwise affected by subsequent resale, repair or replacement of the product.

- The regulations of UN sales law do not apply. The legal warranty of the seller remains unaffected by our guarantee conditions. Within the maximum scope of applicable mandatory laws, these guarantee conditions are your only and exclusive remedy and apply instead of any other expressed or implicit guarantee conditions. **Promed** shall not be liable for exceptional, incidental, punitive or consequential damage including, but not restricted to loss of profit, downtime, loss of revenue, costs for replacement equipment or facilities, insurance claims by third parties, violations of property due to purchase or use of the product or guarantee violation, violation of a contract, gross negligence, product defects or other legal or statutory conditions, even if **Promed** was aware of the possibility of such damage. **Promed** shall not be liable for any delay in claims under guarantee.
- **Promed** shall not be liable for any translation errors.

For prompt processing, the following information must be provided:

1. Original purchase receipt/invoice or dealer stamp with purchase date
2. Identified defect
3. Device designation / type / serial/batch number

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- EN** | Promed GmbH · Lindenweg 11 · D-82490 Farchant · Tel: +49 (0)8821/9621-0
- FR** | Promed GmbH · Lindenweg 11 · D-82490 Farchant · Tel: +49 (0)8821/9621-0
- IT** | Promed GmbH · Lindenweg 11 · D-82490 Farchant · Tel: +49 (0)8821/9621-0
- ES** | Promed GmbH · Lindenweg 11 · D-82490 Farchant · Tel: +49 (0)8821/9621-0
- NL** | Promed GmbH · Lindenweg 11 · D-82490 Farchant · Tel: +49 (0)8821/9621-0
- PT** | Promed GmbH · Lindenweg 11 · D-82490 Farchant · Tel: +49 (0)8821/9621-0